

MyWam

Indywidualnie I Kompleksowo

Pegaz 

Operation and
maintenance manual

CE

Pegaz - a modern stroller
of excellent driveability

1. The Purpose of the Stroller	3
2. Important Safety Rules	3
3. The Purpose of the Manual	6
4. Parts List	7
5. Accessories.....	9
6. Anti-Corrosion Protection	12
7. Reuse of the Stroller	12
8. The manual	12
9. The Usage of the Stroller	25
10. Cleaning and Maintenance	26
11. Periodical Inspection	27
12. The Stroller’s Technical Data	28
13. Notes	29
14. The Manufacturer	30
15. Declaration of CE Manufacturer’s Conformity	31
16. Warranty Card	32



CAUTION! Before the stroller is used, the user is obliged to read the instructions carefully and follow them closely.

1. The Purpose of the Stroller

The specialistic Stroller Pegaz was designed for children who suffer from neurological and orthopedic problems, which make it hard or impossible for them to move independently. The stroller is equipped with adjustments which make it possible to safely and comfortably adapt it to the child's individual needs.

The stroller Pegaz is for users who are 80-140 cm high and weight up to 30 kg.

Statistically, children from the age of 2 to 10 fit these specifications.

While choosing the stroller it is most important, however, to consider the individual physical features of the child as well as his/her disability.

Follow the instructions of the specialist while using the stroller.

The risk associated with the use of the stroller was described in detail in point 2: "Important Safety Rules."

1.1. The Range of the Stroller's Use

The stroller is suitable for use both inside (house, apartments and health centres) and outside on hard and smooth surfaces.

2. Important Safety Rules

2.1. CAUTION! Before you use the stroller, make sure that it has been assembled properly, and all the mechanisms, fixing elements and fastenings have been carefully assembled, locked and prepared for use. Pay particular attention to the frame folding mechanism as well as backrest assembly and proper installation of all four wheels.

2.2. CAUTION! Never leave your child unattended in the stroller.

2.3. CAUTION! Failure to follow the instructions, particularly those concerning the movable parts of the stroller may be dangerous for the user. Always read the manual before using the stroller.

2.4. CAUTION! To avoid injuries, keep your child in a safe distance from the stroller while folding or unfolding the product.

2.5. CAUTION! Always use restraint systems for the carried children. These include: five-point belts, wedge, groin straps, chest and loin pads, head supports, footrest, safety barrier or tray.

2.6. CAUTION! The stroller must be completely assembled before use. In case of any problems at this point, please contact the Seller and Manufacturer.

2.7. CAUTION! The product is not suitable for running or skating.

2.8. The stroller should be cleaned and maintained regularly. No abrasive materials should be used for cleaning.

2.9. The product contains small parts, which may be dangerous if swallowed by children.

2.10. DO NOT use the stroller if it was damaged or if you suspect it was damaged or if it does not work properly. Contact the Seller and Manufacturer promptly to get advice.

2.11. Make sure you use only the original stroller accessories.

- 2.12. The brake must be locked while taking the child in and out of the stroller.
- 2.13. All additional bags or parcels may influence the stroller's stability.
- 2.14. All adjustments should be performed under the supervision and care of a doctor or a physical therapist.
- 2.15. Only the spare parts provided or approved by the Manufacturer can be used in the stroller.
- 2.16. Any repairs which are not performed by the authorised Manufacturer's service team, result in the loss of warranty. The Manufacturer is not responsible for any bodily injuries or product damages resulting from the repairs performed by persons other than the authorised Manufacturer's service team.
- 2.17. The manufacturer is responsible for the proper functioning of the stroller only if the stroller was bought in a specialist medical store.



2.18. **CAUTION!** Before the stroller is used, the user is obliged to read the instructions carefully and follow them closely.

2.19 Before each use :

- 2.19.1.. Check the stroller for any visible signs of damage or other defects that may put your child in danger.
- 2.19.2. Check if the break works properly.
- 2.19.3. Check for the condition of tire treads.
- 2.19.4. Check if the stroller's frame has been folded properly. The frame should be locked securely.
- 2.19.5. Adjust the stroller's accessories and equipment to the size and needs of the child, e.g., adjust the angle of the seat, the backrest and the footrest, as well as the length of the 5-point belt.
- 2.19.6. After placing your child in the stroller, mount the safety barrier or the tray.
- 2.19.7. Secure your child using all the available restraint systems such as: 5-point belts, wedge, groin straps, chest and loin pads, head supports, footrest, safety barrier or tray. Their aim is to prevent your child from falling or sliding out of the stroller. They are also to prevent your child's feet from sliding off the footrest.
- 2.20. The stroller CANNOT be used for a purpose other than intended.
- 2.21. Never leave your child unattended in the stroller, especially on a hilly surface.
- 2.22. It is not allowed to leave a child in the stroller without the supervision of an adult and responsible carer, especially if the stroller is left in a sloping area.
- 2.23. Never use the stroller while ascending or descending stairs, no matter if the child is in or out of the stroller.
- 2.24. Make sure that neither the child or any other person stands on the footrest while getting in and off the stroller.

- 2.25. It is not allowed to carry a child in the stroller while using any form of transportation such as a car, a bus or a train.
- 2.26. Do not use the stroller's seat as a car seat.
- 2.27. The stroller's brake must be locked while taking the child in and out of the stroller.
- 2.28. The maximum weight for the bag placed on the handle bar is 2 kg.
- 2.29. Make sure your child's limbs do not get in the wheel spokes or other parts of the stroller while using it.
- 2.30. Be especially careful while using the stroller on an uneven or hilly surface, as its center of gravity is likely to shift.
- 2.31. While transporting the stroller e.g., by car, by bus or train, the child **MUST** be removed from the stroller. The stroller needs to be adequately protected from damage during transportation.
The Manufacturer **CANNOT** be held responsible for any damage resulting from improper protection of the stroller during transport.
- 2.32. Never leave your child in the stroller while lifting it or carrying it.
- 2.33. While lifting and carrying the stroller, you need to hold it by the frame or other fixed elements. When folded, the stroller has a special handle that enables easy lifting and carrying. Hold the seat exclusively by the fixed elements of the stroller.
- 2.34. The child's maximum allowed weight is 35 kg. The maximum weight of the storage bag is 2 kg. The maximum load for the anti-slip therapeutic tray is 3 kg. The maximum load for the shopping basket is 3 kg.



2.35. **CAUTION!** Both prior to and during use of the stroller, read the manual instructions and safety rules carefully and follow them closely.



2.36. **CAUTION!** You need to exert caution when using, maintaining or folding the stroller due to the fact that the dimensions of the fixed and moveable slots and holes may pose some danger.



2.37. **CAUTION!** The upholstery must be dry while using the stroller.



2.38. **CAUTION!** The Manufacturer is ONLY responsible for the defects and technical damage resulting from the use of the stroller that is in strict compliance with the instructions and safety rules specified in this manual.



2.39. **CAUTION!** Never leave your child in the stroller while making any alterations, modifications or adjustments. Failing to follow this instruction may result in serious danger to the child or others.

There is a risk of confining and/or squeezing the user of the stroller or the accompanying person while using or adjusting the stroller. You need to adjust the stroller with utmost care. After adjusting the stroller, you need to stabilize the stroller by carefully tightening the knobs/screws. When adjusting the stroller, the child must be removed from the stroller!

3. The purpose of the manual.

The manual contains basic information about the proper use of the Pegaz stroller, its assembly, maintenance and cleaning and warranty conditions. Please keep the manual in an easily accessible place. The manual is intended for the caregivers of special needs children, as well as the doctors and physical therapists who will use the stroller.

IMPORTANT! Keep the manual for future reference.

4. Parts list

4.1 General construction of the stroller



Picture 4.1.

4.2. The stroller consists of:

- 4.2.1. Adjustable, ergonomic pushbars (Rys.4.1. - nr 10)
- 4.2.2 Direction lock for front wheels (Rys.4.1. - nr 8)
- 4.2.3. Shock absorption system for front wheel forks (Rys.8.6a)
- 4.2.4. Shock absorption system for rear wheels (Rys.8.7)
- 4.2.5. Rear wheel brakes (Rys.4.1. - nr 17)
- 4.2.6. Shopping basket (Rys.4.1. - nr 16)
- 4.2.7. Transport lock (Rys.8.9)
- 4.2.8. Frame folding mechanism (upper and lower) (Rys.8.2a-8.2b)
- 4.2.9. Backrest height adjustment mechanism (Rys.8.11a-8.11b)
- 4.3.10. Three types of upholstery covering the seat: (Rys.4.1. - nr 5)
- 4.3.11. Anti-slip therapeutic tray (Rys.8.13)
- 4.3.12. Safety barrier (Rys.4.1. - nr 2)
- 4.3.13. Five-point belts (Rys.4.1. - nr 12)
- 4.3.14. Groin straps (Rys.8.16)
- 4.3.15. Wedge (Rys.4.1. - nr 4)
- 4.3.16. Adjustable head support ↑ (Rys.4.1. - nr 9)
- 4.3.17. Adjustable chest and loin pads (Rys.4.1. - nr 14)
- 4.3.18. Adjustable canopy with a peek-a-boo window (Rys.4.1. - nr 20)
- 4.3.19. Adjustable footrest (Rys.4.1. - nr 7)
- 4.3.20. Leg slipsack (Rys.8.22)
- 4.3.21. Sun umbrella (Rys.8.23)
- 4.3.22. Big storage bag (Rys.8.24)
- 4.3.23. Mosquito net (Rys.8.25)
- 4.3.24. Rain cape (Rys.8.26)
- 4.3.25. Optional inclination bar (Fig. 8.7.2
& Fig.4.1. - nr. 18)

5. Accessories

The Pegaz stroller offers a wide range of accessories.

5.1. Standard accessories. These include:



Fig. 5.1.1.
The back wheels breaks



Fig. 5.1.2.
Shopping basket
(fastened under the seat)



Fig. 5.1.3.
Anti-slip therapeutic tray
(Fastened in the place of the barrier)



Fig. 5.1.4.
Safety barrier (fastened in
the place of the tray)



Fig. 5.1.5.
Five-point belts



Fig. 5.1.6.
Groin straps



Fig. 5.1.7.
Wedge



Fig. 5.1.8.
headrest



Fig. 5.1.9.
lumbar pelottes



Fig. 5.1.10.
Adjustable canopy with
a peek-a-boo window



Fig. 5.1.11.
Adjustable footrest



Fig. 5.1.12.
Leg slipsack



Fig. 5.1.13.
Sun umbrella



Fig. 5.2.14.

Adjusting the height of the backrest



Fig. 5.2.15.

Adjusting the seat's depth



Fig. 5.1.16.

Mosquito net
(fastened in the place
of the rain shield)



Fig. 5.1.17.

Rain shield
(Fastened in the place
of the mosquito net)



Fig. 5.1.18.

Big storage bag
(attached to the pushbar)

6. Anti-corrosion Protection

In order to avoid damage resulting from corrosion, the stroller shouldn't be used or kept in wet or damp spaces. If the stroller has been soaked in water, it needs to be dried with a soft cloth and treated by a preservative that has light oil basis. The stroller is made with high quality shell.

If the enamel has been damaged, you need to protect it with the lacquer recommended by our firm. Regular cleaning and maintenance using light oil ensures long and hassle-free use of the stroller.

7. The Reuse of the stroller

The product is suitable for reuse. All products that are reused may change their usability due to change in their specifications and performance characteristics and, therefore, need to be checked for efficacy. The safety of the persons using the stroller as well as the accompanying people may be endangered in such circumstances. When maintained properly, the product keeps its quality for about three years. To reuse the stroller you need to dismantle all upholstered parts and exchange them for the new ones.

The stroller's frame, the wheels and the handles need to be cleaned and disinfected with available cleaning agents. The stroller needs to be subsequently examined by an authorized specialist as to wear and tear as well as damage. All parts that are mismatched or unsuitable for use need to be replaced. Only then the stroller can be reused.

The maintenance plan and information about the parts and tools are contained in part 11: "The periodical maintenance."

8. The manual

8.1 Mounting

8.1.1. Stroller unfolding. To unfold the stroller: release the transport lock and pull up the handlebar until it is fully unfolded. The stroller's frame is properly set up only when the "expanding" points behind the backrest and footrest are properly locked. Both locking mechanisms should be pushed down until they are blocked.



Fig. 8.1.1a.



Fig. 8.1.1b.

8.2. Folding the stroller

In order to fold the stroller, remove all stroller accessories.

Next, lift the stroller's two frame folding mechanisms and fold the stroller like a 'pocket knife'.



Fig. 8.2a.



Fig. 8.2b.

8.3. Adjustable, ergonomic handlebars

To change the handlebars' angle, press the buttons on their tops and turn.

After reaching the desired angle, stop pressing the button and the handle will stay in this position. The handles are adjustable every 90 degrees.

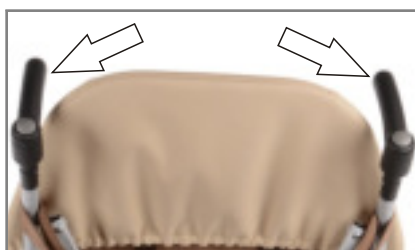


Fig. 8.3.

8.4. Front wheel direction block.

To prevent the front wheels from turning, press the button between the front wheels.

To unblock the front wheels, lift the pushbutton between the front wheels.



Fig. 8.4.a.



Fig. 8.4.b.

8.5. Front wheel forks shock absorption system

The stroller is equipped with the front wheel absorption system that works automatically. It should not be adjusted.



Fig. 8.5.

8.6. Back wheel forks shock absorption system

The stroller is equipped with the front wheel absorption system that works automatically. It should not be adjusted.

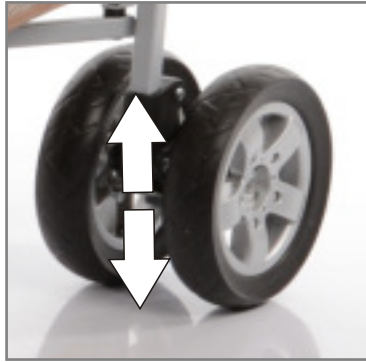


Fig. 8.6.a.

8.7. Back Wheel Brakes and optional inclination bar

To put the break on, press the break pedal down on each back wheel.

To switch the brake off, pull the brake pedal up. Fig. 8.7.1.

To lift the front wheels, simultaneously press the inclination bar and pull the handlebars towards you. Fig. 8.7.2.

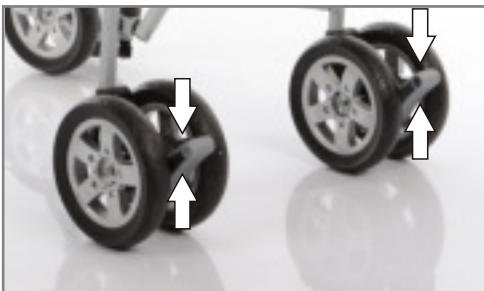


Fig. 8.7.1.



Fig. 8.7.2.

8.8. Shopping basket (fastened under the seat)

The stroller is equipped with a shopping basket, fastened to the lower part of the frame. The basket is used to carry small things used in everyday life, resistant to shock, hits, breakage, water and dirt. Maximum load for the basket is 3 kg.



Fig. 8.8.

8.9. Transport lock

The stroller is equipped with a transport lock, used to protect the folded (packed) frame against its self-opening during the transport. The transport lock should be engaged when folding the frame, and removed when unfolding the frame. The frame should not be unfolded when locked or it may be damaged.



Fig. 8.9.

8.10. Frame folding mechanisms

The two folding mechanisms are very important elements of the stroller. They are situated behind the backrest and footrest. Proper use of the mechanism is important for the safety of the child and others. The exact description of the proper use of the frame folding mechanism is part of the description of the stroller's assembly.

Always pay attention to things like damage to the stroller's frame or its proper operation.

If you have any doubts as to the stroller's proper operation, do not operate it as it may put a child or others in danger.



Fig. 8.10.

8.11. Adjusting the stroller's backrest's height

The stroller is equipped with a backrest that can be enlarged.

To adjust the height of the backrest, remove the upper part of the upholstery and put the prolonging pipes into the slots in the corners.

Then, fasten the prolonging parts with Velcro found at the top of the upholstery.

CAUTION! When folding the stroller, you need to remove the prolonging parts from the backrest.

CAUTION! Remove the child from the stroller before doing any adjustments.

Not following these safety rules puts the child in danger!



Fig. 8.11a.



Fig. 8.11b.

8.12. Mounting the tray or the barrier

The tray/barrier is mounted through clicking the tray/barrier clamps on the stroller frame.



Fig. 8.12.

8.13. Anti-slip therapeutic tray (locked into the safety barrier attachment tubes in place of the safety barrier)

The stroller is equipped with an anti-slip therapeutic tray whose maximum load is 3 kg. Slide it into the attachment tubes and listen for the 'click'.

The clearly audible 'click' is an indication that the tray has been securely attached to the seat.

To take the tray out, press the two buttons on the attachment tube and slide the tray out.

CAUTION! You cannot use the stroller without either the therapeutic tray or the safety barrier!



Fig. 8.13.

8.14. Safety barrier (locked into the safety barrier attachment tubes in place of the tray)

Slide the safety barrier into the attachment tubes and listen for the 'click'.

The clearly audible 'click' is an indication that the safety barrier has been securely attached to the seat.

To take the safety barrier out, press the two buttons on the attachment tube and slide the safety barrier out.

CAUTION! You cannot use the stroller without either the therapeutic tray or the safety barrier.



Fig. 8.14.

8.15. Five-point belts

To apply the five-point belts, put on the straps on the right and left side of your child.

Then, insert both straps into the strap clamp located in the seating unit.

To adapt the strap length, move the belt adjusters up or down and put on the covers.

CAUTION! Any adjustments of the stroller and its parts must be performed WHEN THE CHILD IS NOT in the stroller. Failure to follow the instruction may cause serious danger for the carried child!



Fig. 8.15.

8.16. Groin Straps

To install or uninstall the groin straps, fasten or unfasten the velcro at the side of the backrest and the seat.



Fig. 8.16.

8.17. The wedge.

To assemble the wedge, fasten it with a Velcro from the front or the back of the skirt, and wrap it with a band to secure it.



Fig. 8.17a.



Fig. 8.17b.

8.18. Adjustable head support - ⬆️⬆️

To adjust the height of head support, place the head support at the desired height using velcro and put the fastenings through the holes and fasten behind the backrest.

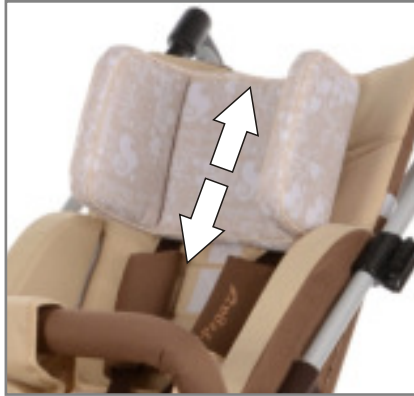


Fig. 8.18.

8.19. Chest and loin pads

To mount or dismantle the chest and loin pads, fasten or detach the hook and loop fasteners at the edge of the backrest and the seat, where the fasteners are sewed.



Fig. 8.19.a.



Fig. 8.19.b.

8.20. Adjustable canopy with a peek-a-boo window

To unfold or fold the canopy, pull its hood in the right direction and set the desired position. The window is opened and closed with a zipper.



Fig. 8.20.

8.21. Adjustable footrest

To assemble the footrest, put it perpendicular to the stroller's frame in the place where the fastening pins stick out from the frame. Then, press the footrest to lock it. After mounting the footrest, make sure it is properly secured.

To check if that is the case, pull it up. If it doesn't unclip, it is mounted correctly.

To dismantle the footrest, press the two buttons on both sides of the footrest and pull the footrest up.

CAUTION! You cannot use the stroller without the footrest.



Fig. 8.21.

8.22. Leg slingsack

To mount the leg slingsack, put its lower part on the footrest and wrap its upper part around the safety barrier and fasten it to the velcro.



Fig. 8.22.

8.23. Sun umbrella

To mount the umbrella, fasten the umbrella holder to the stroller frame and slide the umbrella in the holder.



Fig. 8.23.

8.24. Storage bag (fastened on the handlebar)

The big bag is fastened onto the handlebars using two velcros situated on the bag's strap. The bag's maximum load is 2 kilos.

CAUTION! Exceeding the bag's maximum load impacts the stability of the stroller and puts the child and others in danger.



Fig. 8.24.

8.25. Mosquito net (fastened in place of the rain shield)

To mount the mosquito net, pull it on the canopy and the footrest.



Fig. 8.25.

8.26. The rain shield (fastened in the place of the mosquito net)

To mount the rain shield, pull it on the canopy and the footrest.



Fig. 8.26.

9. The Usage of the Stroller

9.1. How to prepare a child / patient / stroller's user

9.1.1. Make sure that the child wears comfortable footwear that stabilizes the ankle joint.

9.1.2. To prevent abrasions and because of hygienic reasons, make sure the child wears clothes so that the skin does not touch the upholstery.

9.1.3. In case hep locks, drains, bags, catheters etc. are used, make sure they do not hinder the operation of the stroller and they do not fall out, tear out or tighten.

9.2. Use manual

9.2.1 Before you use the stroller you need to do each of the following:

9.2.1.1. Check the stroller and all its parts for good technical condition.

Caution! Do not use the stroller if its technical condition is questionable or in case you have doubts as to its condition. Keep the safety of your child as well as other people's in mind at all times.

9.2.1.2. Place the stroller on a smooth and safe surface and assemble it properly. Use the instruction manual.

9.2.1.3. Adjust all parts of the stroller including the seat, the backrest, the head support, the chest and loin pads, the footrest, the handlebar, the canopy and adjust the length of five-point belts and rear wheel shock absorption system.

9.2.1.4 Assemble the safety barrier and the anti-slip therapeutic tray.

9.2.1.5. Prepare the five-point belts, the groin straps and the footrest for assembly.

9.2.1.6. Switch the central brake on.

9.2.1.7. Place the child in the stroller and ensure that all previously made adjustments are correct and safe.

9.2.1.8. Fasten the five-point belts, the groin straps and the footrest straps on your child.

9.2.1.9. Attach the handlebar, the footrest and other necessary elements.

9.2.2. If the child wets himself/herself, put a special waterproof pad on the seat. The pad can be purchased in pharmacies or medical stores.

9.2.3. Never let the stroller roll on its own, always use the handlebar.

9.2.4. When riding on slopes or ramps, always assume the forward-facing position of the stroller.

9.2.5. While riding on small bumps like low doorsteps or curb, etc., push the handlebar down, lift the stroller and place the front wheels on the bump. Then, pull the handlebar up, lift the rear wheels and place them on the surface.

10. Cleaning and maintenance

10.1. Small spills and stains can be removed with a kitchen sponge and clean, warm water.

10.2. Stains should be removed immediately.

10.3. Do not wipe or rub the surface. Rubbing will result in the spreading of the stain and can destroy the surface of the fabric.

10.4. If water, food, beverages or plant oil have stained the fabric, do the following:

10.4.1. Dry the stain by placing a white paper towel, a tissue, or a clean absorbent cloth on the entire surface of the stain.

10.4.2. Spot clean the stains only: moisten a small piece of cloth with a diluted detergent and warm water (in case of blood stains, use cold water instead), and remove the stain moving from the outer edge of the stain towards the center to prevent spreading. Do not soak the stain with water, as it may cause rims after drying. Excessive wetness may also enlarge the stain or destroy the foam inside the upholstery.

10.4.3. You may have to repeat the above actions (10.4.2.) until the stain is completely removed. Remember to always remove the excess moisture (10.4.1.) before each subsequent wiping.

10.4.4. Finally, rinse the stain using a clean sponge or cloth soaked with lukewarm water. Do not use detergent at this point. Wipe from the outside of the stain to the center.

10.5. Disinfectants that do not contain organic solvents can be used in accordance with the specifications given by the product manufacturer.

10.6. Do not submerge the fabric.

10.7. Do not use organic solvent or dry-cleaning detergents.

10.8. To disinfect the surface, use common, every-day bleach diluted in water (maximum bleach concentration should be 3%).

10.9. Make sure the product is dry before you use it.

10.10. The stroller should be kept in a clean, dry, well-ventilated room.

10.11. **CAUTION!** The manufacturer is not held responsible for the damage resulting from using inappropriate cleaning products.

11. Periodical maintenance checks.

In order to use the stroller without failure, you need to regularly check it and service it.

If the user of the stroller cannot service it himself/herself, the stroller should be serviced at the technical service point or from the manufacturer, who is obliged to check it and service it. There may be charge for the aforementioned service.

All repairs must be done at technical service points using original spare parts

The periodical service:

WHEN?	WHAT?	REMARK
Before starting out		
	Test brakes for faultless operation	Activate brake pedal(-s) to the limit. The locked wheels can not be able to turn under operating conditions
	Check all construction's elements	Wheels, frame, seat, 5 point belts, footrest, safety barrier
	Check air pressure of the tyre and tyre's profil	For this use an air pressure gauge. Carry out visual check yourself.
	Check the back tubes and frame tubes for firmness	If deformations or cracks occur in the welding seams, contact an specialist workshop immediately for repairs

Every 1 month		
	Lubricate the following components with a few drops of oil – Moving parts of the locking mechanism	Components must be free from used oil residues before lubrication
	Check all screw connections for secure fit	Yourself or with the aid of a helper
Every 6 months		
	Cleanness	Yourself or with the aid of a helper
	General condition	Yourself or with the aid of a helper

12. The Stroller's Technical Data

PARAMETER	PEGAZ
Length	103 cm
Width	55 cm
Height	118 cm
The seat's width	30 cm
The seat's depth	24 & 28 cm
The backrest's height	50 & 60 cm
The distance of the footrest from the seat	18-42 cm, 3 points
The seat's inclination	28 degrees
The angle between the seat and the backrest	100 degrees,
The size of the front/rear wheels	18 cm x 4 cm/18 cm x 4 cm
The front/rear tire pressure	---

12.1. Material list

Dry-sprayed steel and aluminum

Fabric: 100% polyester

Man-made materials: PP,PCV, ABS, PA, PE, PU

12.2. Weight and dimensions

Stroller PEGAZ	Width	Height	Lengh	Weight
Unfolded with accesories	55 cm	118 cm	103 cm	14 kg
Folded and packaged	36 cm	46 cm	133 cm	17 kg

Table: Weight and dimensions

12.3. Table of parameters

Child's maximum weight	30 kg
Child's maximum chest width	35 cm
Minimum and maximum lower leg lengh	19 & 43 cm
Minimum thigh lengh	22 & 35 cm
Child's maximum height	140 cm

Table: Child's parameters

12.4. Utilization and recycling

The used stroller needs to be dropped off at a recycling centre following the rules that are in force in a given country.

13. Notes

14. The Manufacturer

MyWam Kupiec, Bartold, Angres
spółka jawna [general partnership]
ul. Szczecińska 10, 41-516 Chorzów, Poland
telephone number : +48 32 733 11 31
www.mywam.pl
e-mail: mywam@mywam.pl

The Pegaz stroller is a polish product!

The authorized dealer:



MyWam sp.j reserves the right to introduce technical and commercial changes to the content of the Operation and maintenance manual without prior notice. Any change shall be considered as the date of latest revision.

Last modifications: June 2016

15. Declaration of CE Manufacturer's Conformity



MYWAM Kupiec, Bartoń, Angres spółka jawna
ul. Szczecińska 10, 41-516 Chorzów, Polska
tel. 0048 32 733 11 33
mywam@mywam.pl



DEKLARACJA ZGODNOŚCI WE

EC DECLARATION OF CONFORMITY

Wytwórca: <i>Manufacturer:</i>	MYWAM Kupiec, Bartoń, Angres spółka jawna ul. Szczecińska 10, 41-516 Chorzów, Poland
Produkt: <i>Product:</i>	Wózek inwalidzki specjalny PEGAZ Specialist Stroller PEGAZ

Deklaruję z pełną odpowiedzialnością, że wózek inwalidzki specjalny Pegaz, będący wyrobem medycznym klasy I, jest zgodny z wymaganiami dyrektywy Rady Europy o wyrobach medycznych - 93/42/EEC z 14 czerwca 1993 roku zmienionej przez 2007/47/EEC z 21 marca 2010 roku, przyjętej w Polsce na mocy ustawy o wyrobach medycznych z dnia 20 maja 2010 r. (Dz. U. nr 107, poz. 679), oraz zgodny z normami zharmonizowanymi:

I solely declare that the specialist Stroller Pegaz is a class I device, and it is in conformity with the requirements of EU Council Directive for Medical Devices 93/42/EEC of 14 June 1993 amended by 2007/47/EEC of 21 March 2010, implemented in Poland under the Medical Devices Act of 20 May 2010 (Dz. U. nr 107, poz. 679), and is in conformity with harmonized standards:

PN-EN 12183:2014	PN-EN 12182:2012	PN-ISO 7176 part 8	ISO 7176 part 1,3,5,7
PN-EN 1021-1:2007	PN-EN 980:2010	PN-EN 1041:2010	PN-EN ISO 14971:2011



Chorzów, 22 czerwiec 2016 r. / 22 June 2016

Data/Data

Wojciech Bartoń

Podpis/Signature
MyWam
Kupiec, Bartoń, Angres Spółka Jawna
41-516 Chorzów, ul. Szczecińska 10
www.mywam.pl
tel. 32 733 11 33, e-mail: mywam@mywam.pl
NIP: 527 579 87 03, REGON: 14347969
KRS: 000048471



WARRANTY CARD

Series and number: AH 

date of purchase:

stamp and signature of the Seller:



Warranty terms and conditions

1. This warranty is valid for a period of 24 months from the date of purchase.
2. Any repairs needed within the 24-month period will be performed free of charge.
3. To make a complaint about the product, the purchaser should bring the stroller to the seller or manufacturer in its original packaging together with the properly protected parts.
4. Warranty repairs do not cover the damage caused by inappropriate protection of the stroller and its parts during delivery.
5. The person filing a complaint cannot make claims under the warranty for damage that has been caused by inappropriate labeling or inadequate insurance.
6. The purchaser will be responsible for all costs associated with shipping.
7. The warranty period will be extended by the time needed to make repairs.
8. This warranty excludes any liability other than that expressly stated previously.
9. The purchaser can exchange the stroller if:
 - 9.1. The same defects exist after the stroller has been submitted for repairs twice and the purchaser cannot use the stroller.
10. The purchaser must use the stroller according to its intended use and strictly following the above instructions.
11. This warranty is void if the purchaser failed to use the stroller according to its intended or appropriate use, or if s/he didn't follow the instructions.
12. This warranty is void if the purchaser made repairs or modified the product.
13. This warranty is void if the product has been damaged as a result of misuse, inappropriate maintenance, inappropriate storage, mechanical damage or regular wear and tear.
14. This warranty is void if a third party has carried out repairs or modifications. It is not advisable to have the stroller repaired by an unauthorized company.
15. The warranty card is valid if it contains the date of purchase confirmed by the Seller's stamp and signature as well as the type and serial number of the stroller.
16. The warranty card is void if it contains corrections or has been crossed out.
17. The Manufacturer offers paid, post-warranty service.
18. Any matters that have not been mentioned in the warranty card are governed by the provisions of the Civil Code.
19. All complaints should be directed to the Seller or the Manufacturer.