



CE



## Operation and maintenance manual

Edition 3 of 2020-12-18

### Special stroller Pegaz

Umbrella type stroller



**ATTENTION!** Users with sight, reading or cognitive impairment are requested to report to the Manufacturer in order to obtain information for the user in an appropriate format.

### **Purpose of the Manual**

The operation and maintenance manual contains basic information necessary for the correct use of the Pegaz stroller, including the principles of safe use, description of its construction, operation and adjustment, preparation of the stroller for operation, how to maintain and clean the stroller, and warranty conditions. It is designed for persons who directly care of children with disabilities (assistants child - parents, guardians), which will be transported in a stroller. It is also intended for doctors and physiotherapists operating the stroller..

**ATTENTION!** Keep this manual for future reference!

Keep the manual in an easily accessible place.

Manufacturer's contact details:

- Postal address: MyWam sp.j., ul. Szczecińska 10, 41-516 Chorzów, Poland
- contact phone number: +48 32 733 11 31
- e-mail address: [mywam@mywam.pl](mailto:mywam@mywam.pl)



**ATTENTION!** Before using the stroller, each User is obliged to read and follow this manual.

**ATTENTION!** This manual is an integral part of the product and must always be attached to the product being sold.

User and maintenance manual version: EN

Edition: third of: 2020-12-18
-------------------------------

MyWam Kupiec, Bartold, Angres sp.j. reserves the right to make technical and commercial changes to the content of the Operation and Maintenance Manual without prior notice. Each change will be determined by the date the instructions were last updated .

## Table of Contents

▪ Introduction.....	5
1. Purpose of the Pegaz stroller .....	5
2. Basic information about the Pegaz stroller .....	5
3. Quick start .....	6
▪ Safety.....	7
4. Important safety instructions and rules .....	7
5. Nameplate .....	13
6. Warning labels.....	13
7. Symbols designation.....	13
▪ Product .....	14
8. General construction of the stroller .....	14
9. Configuration and use - stroller frame .....	15
9.1. Unfolding the stroller.....	15
9.2. Folding the stroller.....	15
9.3. Front double wheels (solid wheels). Traveling direction lock function. ....	16
9.4. Rear double wheels (solid wheels). Independent brakes system. ....	16
9.5. Amortisation system for all four double wheels .....	16
9.6. Tilting foot.....	17
9.7. Adjustable ergonomic handles to push the stroller .....	17
9.8. Footrest. Regulation and use.....	17
9.9. Seat depth regulation. ....	18
9.10. Backrest height adjustment .....	18
9.11. Backrest tilt angle adjustment .....	19
10. Standard equipment .....	20
10.1. Five points safety belts .....	20
10.2. Transport lock .....	20
10.3. Safety barrier.....	21
10.4. Shopping basket.....	21
10.5. Wedge .....	21
10.6. Abduction and stabilizing belts .....	22
11. Additional equipment .....	22
11.1. Adjustable canopy with window.....	22
11.2. Therapeutic table .....	23
11.3. Loin cushion .....	23
11.4. Headrest.....	23
11.5. Narrowing pads.....	23
11.6. Tourist bag .....	24
11.7. Raincover.....	24
11.8. Mosquito net.....	24
11.9. Sleeping bag .....	24
11.10. Sunshade .....	24
12. Technical specification of Pegaz stroller.....	26

12.1.	User parameters .....	26
12.2.	Material composition of the stroller .....	26
13.	Delivery and tools .....	26
14.	Assembly and disassembly of the stroller.....	27
15.	Preparation for use .....	28
16.	Using the stroller .....	28
16.1.	Carrying the stroller.....	28
16.2.	Getting on and off the stroller.....	29
16.3.	Correct position in a stroller.....	29
16.4.	Riding in a stroller.....	29
16.5.	Moving on slopes.....	29
16.6.	Overcoming thresholds or curbs .....	29
16.7.	Transport in a car and other vehicles.....	29
16.8.	Cleaning and care .....	30
17.	Storage of the stroller .....	30
17.1.	Preparation of the stroller for long-term storage.....	30
17.2.	Corrosion protection .....	30
18.	Re-use.....	31
19.	Service and periodic inspections.....	31
20.	Disposal .....	32
21.	Compliance with requirements for medical devices .....	32
22.	Notes .....	34
23.	List of authorized service points .....	35
24.	Warranty Card.....	36

## ▪ Introduction

Thank you for purchasing special Pegaz Special Stroller. We hope that it will be the cause of many wonderful moments with your child and will be helpful every day. We make every effort to ensure that our products meet your expectations and the highest standards. We encourage you to contact our employees and follow us on our website [www.mywam.eu](http://www.mywam.eu) and on social media. There you can find out many interesting things about our products.

### 1. Purpose of the Pegaz stroller

- The Pegaz Special Stroller is designed to transport a child with neurological, orthopedic or other conditions that hinder or prevent him from moving independently.
- The Pegaz Special Stroller is a medical device intended for transporting a child with four-limb, three-limb, one-sided or crossed limb paralysis or paresis. Among the disease entities of a particular child, among others, there can be mentioned such diseases as: cerebral palsy, meningeal hernia, condition after spinal injury, condition after craniocerebral trauma, condition after stroke, paralysis of various origin and other dysfunctions damaging the locomotor and preventing independent movement of the child.
- The Pegaz Special Stroller has been designed to carry one child.
- The Pegaz Stroller, thanks to many regulations, allows comfortable and safe adaptation to the individual needs of the child.
- Pegaz stroller is designed to carry a child whose weight does **not exceed 40 kg, height from 80 cm to 140 cm**. Statistically, they are children from **2 to 10 years old**.
- When choosing a stroller, however, the most important are the individual physical and mental characteristics of the child, its dimensions and weight, housing conditions, environment and, above all, the individual's illness.
- Use the stroller as directed by a specialist.
- The Pegaz stroller is intended for both indoor (e.g. apartment, rehabilitation center) and outdoor (e.g. park, pavement), on hard and level ground, on which all four wheels touch the ground and their contact with the ground is sufficient to properly drive the wheels.
- The Yeti stroller is intended to be controlled and propelled only by an assistant (parent, guardian).
- The user of the Yeti stroller is both the child who will be transported in it and the child's assistant (his parent, guardian, doctor, physiotherapist) who will transport the child in the stroller and look after him.
- Pegaz stroller with proper care and maintenance maintains its quality for a period of 3 years and this is the expected period of its use.
- The risks associated with using the stroller are described in detail in the chapter: "Important instructions and safety rules".

### 2. Basic information about the Pegaz stroller

The Pegaz wheelchair has many options, e.g.

- backrest tilt angle adjustment ..... see poin 9.11
- seat depth adjustment..... see poin 9.9
- backrest height adjustment – extensions.....see poin pkt 9.10

- footrest adjustment..... see poin 9.8
- narrowing inserts ..... see poin 11.5
- additional equipment ..... see poin 11

### 3. Quick start

The advantage of the Pegaz special pram is its ease of use and compactness. It can be prepared for use in a few simple steps (Photo 3.1):

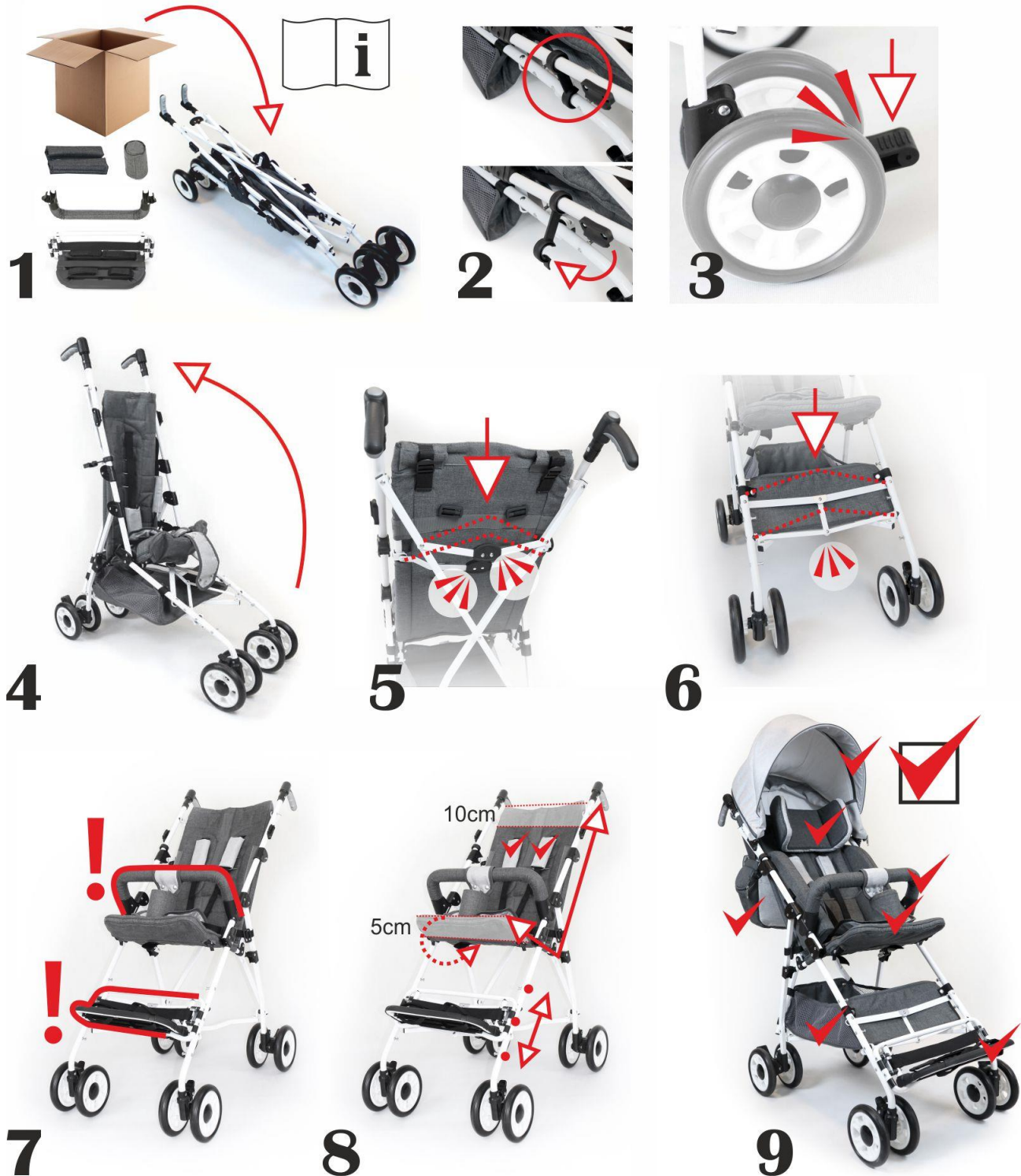











Photo 3.1



▪ **Safety**

#### 4. Important safety instructions and rules

General tips:

-  • **WARNING!** Risk of accident - before using the stroller, the user is obliged to read and follow the entire contents of this manual.
-  • **WARNING!** Risk of accident - Before starting and while using the stroller, it is essential to comply with the safety instructions and rules described in this manual, and comply with the other instructions described in this manual.
-  • **WARNING!** Failure to follow the manual, in particular those regarding the moving parts of the stroller, may be dangerous for the user.
  - Always read the operating and maintenance manual before using the stroller.
-  • **WARNING!** Never leave your child unattended in a stroller, as there is a risk of injury and there is a risk of accident!
-  • **WARNING!** The product contains small parts that can be dangerous if swallowed by a child!
-  • **WARNING!** Risk of injury - always use all available security systems for the child using a stroller. These include: five point belts, narrowing inserts, abduction and stabilizing belts, wedge, safety barrier or in its place a therapeutic table, footrest and straps for placing feet on the footrest.
-  • **WARNING!** There is a risk of burns when using the stroller - be careful when transporting a child in a stroller at high and low temperatures and in contact with the skin, e.g. in the sun, in cold weather, in a sauna, because the temperature of the stroller surface may rise when the stroller is exposed to solar radiation or the stroller surface temperature may drop when exposed stroller for frost.
-  • **WARNING!** When using the stroller or operating the stroller or folding the stroller, there may be a risk to the health of users and people in the immediate vicinity, caused by the dimensions of fixed and movable gaps and openings. Be particularly careful when doing this.
-  • **WARNING!** When using and operating the stroller, as well as when folding and adjusting the mechanisms, there may be a risk of entrapment and squeezing of parts of the body of the user or accompanying person in the openings and gaps between elements.
  - Be particularly careful when doing these operations.
  - After completing these operations, stabilize the positions of the individual components of the stroller by tightening the nuts, knobs and screws thoroughly. Each time you perform the above steps, you must first pull the child out of the stroller!
  - **ATTENTION!** All adjustments to the stroller and its components must be made **WITHOUT** a CHILD in the stroller! Failure to comply with the above warnings poses a serious danger to the child using the device!
  - **ATTENTION!** Never exceed the maximum load.
  - **ATTENTION!** For Pegaz stroller with load the maximum total load amounts to 40 kg. This value should not be exceeded.
  - **ATTENTION!** The maximum load of the travel bag is 2 kg.
  - **ATTENTION!** The maximum load of the shopping basket is 2 kg.




- ATTENTION! The maximum therapeutic table load is 3 kg.
- ATTENTION! Depending on the adjustments of the Yeti stroller - it concerns the position of the push handles, backrest height adjustment, backrest angle adjustment, seat depth adjustment, footrest height adjustment - this stroller may exceed the recommended maximum dimensions.
- ATTENTION! In the event of any medical incident related to the presence of a Pegaz stroller, you must immediately notify the Manufacturer in writing by contacting via registered mail or email with confirmation of receipt using the addresses below:

Postal address: MyWam sp.j., ul. Szczecińska 10, 41-516 Chorzów, Poland

E-mail address: [mywam@mywam.pl](mailto:mywam@mywam.pl)







- The therapy seat is not suitable for children under 2 years old
- It is forbidden to use the stroller as a ladder or as a base for climbing onto it to reach higher
- The stroller must not be used in a way for which it has not been designed.
- ATTENTION! This stroller is not a toy! Keep out of reach of babies and young children
- Do not apply to the head. Danger of suffocation.

Tips in particular on construction and regulation:





-  **WARNING!** All adjustments of the stroller and its elements, all settings, changes, fastenings and other activities related to matching the stroller should be carried out WITHOUT A CHILD placed in the stroller. Failure to do so may cause serious injury to a child and other people!
-  **WARNING!** Adjustments made in the stroller, made incorrectly or not in accordance with the instructions in this manual, pose a serious danger to the child and other persons!
-  **ATTENTION!** Changing the seat tilt changes the overall center of gravity, which affects the stability of the stroller. Any change in backrest tilt requires five-point belt re-adjustment!
- **ATTENTION!** The brakes are not used to slow down the stroller during movement and should only be used to immobilize the stroller.
- The front steering wheels, if the direction of travel lock is off, may vibrate and lead to a sudden stop of the stroller and its falling forward when the surface is uneven. Therefore, with an uneven surface, it is recommended to block the front direction wheel lock and push the truck at a reduced speed.
- All adjustments and settings to be performed under the supervision and care of a doctor and a physiotherapist.



Tips in particular on construction and regulation:

-  • **WARNING!** To avoid injury, make sure the child is at a safe distance when the stroller is assembled and disassembled.
-  • **WARNING!** The stroller must be fully assembled before use. In case of any problems at this stage, please contact the Seller and Manufacturer urgently.
-  • **WARNING!** When unfolding the stroller, pay special attention to secure all belts.
-  • **WARNING!** It is forbidden to place additional weight on the stroller, which in particular applies to placing additional weight on the back of the backrest e.g. backpack or bag. This can cause the stroller to tip over and cause injury! Does not apply to original additional accessories dedicated to the Pegaz stroller not exceeding the permissible load.
-  • **WARNING!** When assembling and disassembling the stroller, keep a safe distance from moving parts of the stroller, as there is a risk of fingers, hands or other body parts being trapped.
-  • **WARNING!** Do not unfold the stroller frame if the transport lock is in place, as it may damage it.
- All additional components attached to the stroller affect its stability.

Tips in particular regarding preparation for use:

-  • **WARNING!** Before using the stroller, make sure that the stroller is properly assembled and all mechanisms, catches and fasteners are properly assembled, closed and prepared for use. In particular, this applies to: the mechanism for folding the frame, attaching the seat to the frame of the stroller and the installation of all four wheels.
-  • **WARNING!** Before riding the Pegaz stroller, make sure that the child is secured and correctly adjusted and fitted with five-point belts, as there is a risk of injury.
-  • **WARNING!** The frame folding mechanism is a very important element of the construction of the stroller, so each time before using the stroller, pay special attention to the absence of cracks and damage to the folding mechanism and its proper operation. In the event of any suspicion you must not use the stroller, as this may endanger the user, child's assistant and other people! In this situation, it is recommended to urgently contact the Seller and the Manufacturer for advice.
-  • **WARNING!** Correct use of the frame folding mechanism is important for the safety of the child and other people.
- **ATTENTION!** The upholstery must always be dry before using the stroller.
- **ATTENTION!** Before each use of the stroller, the activities listed below must be carried out, and in the event of irregularities the stroller must not be used. In this situation, it is recommended to urgently contact the Seller and the Manufacturer for advice.

**The operations before each use of the stroller include:**



- Check the stroller for visible damage or other defects that could endanger the child during stroller operation.
- Check that the stroller frame has been unfolded correctly and the frame folding mechanism has been correctly closed (note, struts have to be lowered).

- Make sure that the necessary structural components, such as a footrest and a safety barrier or therapy table are installed. It is forbidden to use the Pegaz stroller without installing one of these elements.
  - Check the wheels structure condition of all wheels.
  - Adjust the stroller to the child's needs, including setting the appropriate depth of the therapeutic seat and backrest height, appropriate backrest inclination, setting the appropriate height of the footrest, adjust the appropriate length of five-point belts.
  - Check that all adjustments are properly seated and secured.
  - Check that the stroller upholstery is dry.
  - Secure the child with five-point belts to prevent the child from falling out and sliding out of the stroller.
  - Secure the child with all available stabilizing elements, such as five-point belts, narrowing pads, abduction and stabilization belts, wedge, footrest, footrest straps, safety barrier or therapeutic table. This is to protect the child from falling out and sliding out of the stroller and to protect the child's feet from falling from the footrest.
- ATTENTION! The stroller must not be used if the technical condition of the stroller or a single element of the stroller is bad or if the user has any doubts about it, the stroller is damaged or there is only a suspicion that damage has occurred or the stroller is not working properly. In this situation, it is recommended to urgently contact the Seller and the Manufacturer for advice.





*Information particularly regarding the handling of the stroller*

- ATTENTION! It is forbidden to move or lift the stroller with the child inside.
- ATTENTION! When moving or lifting the stroller, hold it with both hands only by the frame and the fixed parts of the wheelchair.
- ATTENTION! Do not lift the wheelchair by the footrest or safety barrier, therapy table or any other accessories.




Information in particular regarding getting in and out of the stroller and the correct position in the stroller:

-  • **WARNING!** Do not stand on the footrest, as there is a risk of the stroller tipping over, injury to the user and risk of damage to the product.
-  • **WARNING!** Make sure that when getting in or out of the stroller, the child and no other person stand on the footrest, as this may cause damage and injury to the user!
- **ATTENTION!** When placing and removing a child from the stroller, the both brakes must be activated.


Tips in particular regarding stroller driving:

-  • **WARNING!** The stroller must not be released by itself or pushed away, as it may cause an accident.
-  • **WARNING!** This product is not suitable for running or skating.
-  • **WARNING!** Use of the product for sport purposes is prohibited.
-  • **ATTENTION!** Make sure that during use of the stroller, the child's limbs do not get between the wheel spokes and the adjustable elements of the stroller.
- **ATTENTION!** When passing through a door or similar narrowing limiting the direction of the stroller, make sure there is adequate space on both sides of the stroller, which prevents your hand or other body parts from being trapped, as well as preventing damage to the stroller.
- When traveling on public roads, follow The Highway Code.




Tips in particular on moving on slopes:

-  • **WARNING!** It is forbidden to leave a child in the stroller without supervision of an adult and responsible assistant (parent, guardian), which in particular concerns leaving the child in the stroller on a slope, as it may cause an accident and injury to the user!
-  • **WARNING!** When using ramps, do not exceed the maximum load of the ramps, as this may cause injury to users and people around you.
-  • **WARNING!** When using ramps, make sure that the ramp is suitable, that the wheels of the stroller are high enough to overcome the driveway, that the stroller frame does not touch the driveway, and that help should be provided to another person as there is a risk of the stroller falling over, damage to the product and injury to the user.
- **ATTENTION!** Always remember that the location of the stroller's center of gravity changes when overcoming unevenness or slopes. In this case, be extremely cautious and secure the stroller.

Tips in particular on overcoming thresholds or curbs:

-  **WARNING!** It is forbidden to climb or slide the stroller down the stairs, both with the child and the stroller itself, because it may cause an accident and injury to the user!

Information particularly regarding the transport of the stroller in a car and other vehicles:

-  **WARNING!** It is forbidden to transport a child in a stroller and at the same time in any other means of transport, e.g. in a car, on a train, in a tram, on a bus, on a plane, on a ship or other, because it may cause injury.
-  **WARNING!** Due to possible injuries using stroller seat as a car seat is forbidden.
-  **WARNING!** Risk of accident and injury to the user and other people - when transporting the stroller in other means of transport, e.g. in a car, bus, train, make sure that the stroller is correctly fastened to prevent injury to passengers during collision or sudden braking.
- **ATTENTION!** When transporting the stroller in other means of transport, e.g. in a car, on a bus, on a train, the stroller must be transported without a child and the stroller must be well protected against any damage.
- **ATTENTION!** The Manufacturer is not doesn't ake any responsibility for damages resulting from inadequate product protection during transport.
- **ATTENTION!** When transporting the stroller in other means of transport, e.g. in a car, on a bus, on a train, do not use one seat belt to secure both the passenger and the stroller at the same time
- Use the transport lock when transporting the stroller and storing it.

Tips especially on maintenance and service:

- **ATTENTION!** The Manufacturer is responsible for defects or technical damage to the stroller only caused by proper use of the stroller. The responsibility for the proper functioning of the stroller rests with the Manufacturer only if you strictly follow all the Manufacturer's instructions.
- **ATTENTION!** The Manufacturer is responsible for the proper functioning of the stroller only if original parts and original additional accessories are used.
- Only original spare parts that are supplied or recommended by the Manufacturer may be used in the stroller.
- Repairs to the stroller carried out by persons other than the authorized Service (see Chapter 24.) or the Manufacturer will void the warranty. The Manufacturer is not responsible for personal injury or damage to the product that may result from repairs carried out by persons other than the authorized Service center (see Chapter 24.) or the Manufacturer.
- The stroller should be cleaned and looked after regularly. Do not use abrasive cleaning agents.
- The Manufacturer is responsible for the proper functioning of the stroller only if it is purchased in a specialized medical store.
- The Manufacturer is not responsible for the appropriate selection of accessories for the user.

### 5. Nameplate

- The nameplate contains basic information about the product: name, model, serial number (identification), maximum load, date of manufacture, confirmation of product compliance for medical devices, warning about proceeding in accordance with the Operation and Maintenance Manual and manufacturer's data (see Fig. 5.1).
- The nameplate is located on the rear parts of the intersecting of the frame


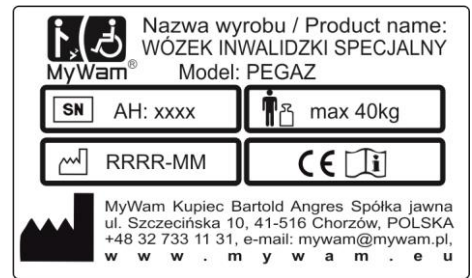


Figure 5.1




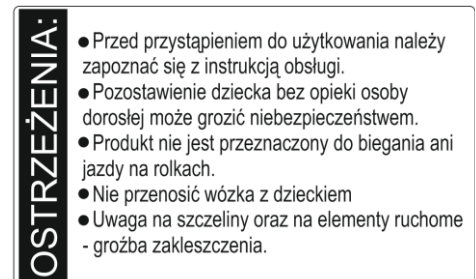


### 6. Warning lables

- The warning label contains important warnings for the user (see Fig. 6.1).
- The warning lable is located on the rear parts of the intersecting of the frame

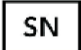

Rys 6.1

### 7. Symbols designation

	Product serial number	 The Manufacturer's name
	Permitted total user weight	DESCRIPTION <b>"WARNING!"</b> and the symbol in this manual are used to enhance the reader's attention to the content marked with this sign. Non-compliance with content marked with this description may endanger the health and safety of the user and accompanying persons, and may also damage the product..
	Date of manufacture	
	Conformity mark according to the Medical Devices Directive No. 93/42 EWG Annex VII	
	Warning about proceeding in accordance with the product Operation and Maintenance Manual	

▪ **Product**

**8. General construction of the stroller**

The Pegaz Special Stroller for Children consists of one part - the frame with the backrest and the seat, which are its integral part. In addition, the stroller has extensive options for additional equipment that can be selected according to the individual needs of the user.

• **Standard equipment**



- 1- Ergonomic handles for pushing the trolley..... see point 9.7
  - 2- Backrest height adjustment mechanism..... see point 9.10
  - 3- Mechanism for adjusting the backrest angle.. see point 9.11
  - 4- Transport lock ..... see point 10.2
  - 5- Upholstery
  - 6- Shopping basket..... see point 10.4
  - 7- Independent brakes on the rear wheels..... see point 9.4
  - 8- Four wheel amortization system ..... see point 9.5
  - 9- Front wheels direction locker ..... see point 9.3
  - 10-Set of 4 double solid gel wheels ..... see point 9.3 i 9.4
  - 11-Adjustable, detachable footrest ..... see point 9.8
  - 12-Seat depth adjustment mechanism ..... see point 9.9
  - 13-Abduction and stabilizing belts \* ..... see point 10.6
  - 14-Wedge \* ..... see point 10.5
  - 15-Safety barrier ..... see point 10.3
  - 16- Five points safety belts..... see point 10.1
- \*) options to choose from as standard equipment

• **Additional equipment**

- 17- Tourist bag..... see point 11.6
- 18- Sleeping bag for legs..... see point 11.9
- 19- Narrowing inserts..... see point 11.5
- 20- Headrest..... see point 11.4
- 21- Sunshade..... see point 11.10
- 22- Adjustable canopy with window .....see point 11.1
- 23- Raincover..... see point 11.7
- 24- Mosquito net..... see point 11.8
- 25- Loin pad ..... see point 11.3
- 26- Therapy table..... see point 11.2





## 9. Configuration and use - stroller frame

### 9.1. Unfolding the stroller

Before unfolding the Pegaz stroller, make sure that all four wheels are on the ground. In order to unfold the stroller, you must first remove the transport lock located on the right side of the stroller. Then, holding the push handles, lift the upper part of the backrest, while gently sliding the handles outward. The next step is to loosen the stroller structure by lowering the struts (located at the front of the stroller at the height of the shopping basket and at the back of the stroller behind the backrest). You should push them down until they reach the horizontal position.

### 9.2. Folding the stroller

To assemble the stroller, first remove all accessories. Next, pull up the struts located between the front wheels at the height of the shopping basket (Photo 9.1.2) and at the back of the wheelchair behind the backrest (Fig. 9.1.3). The next step is to push the handles inwards (Fig. 9.1.4) and to push simultaneously the upper part of the stroller forward and down (Photo 9.1.5). This results in the folding of the stroller. To secure it, fasten the transport lock (Photo 9.1.6) located on the right side of the stroller.



Photo 9.2.1

*Basic position of the strut*

Photo 9.2.2

*Raising the lower strut*

Photo 9.2.3

*Raising the upper strut*

Photo 9.2.4

*Pushing handles inwards*

Photo 9.2.5

*Folding the stroller*

Photo 9.2.6

*Closing transport lock*

### 9.3. Front double wheels (solid wheels). Traveling direction lock function.

The front wheels are an integral part of the frame and can be removed for replacement.

- **Direction lock function.** In the event of uneven surfaces, moving on sloping surfaces, when crossing thresholds or curbs, it is recommended to install the lock and block the turning of the front wheels. In order to block the steering function of the front wheels, the button between the front wheels should be raised (Photo 9.3.1). To re-enable the steering wheel turning function, push it down (Photo 9.3.2) and place the lock in the slot.



Photo 9.3.1

*Direction block on*



Photo 9.3.2

*Direction block off*

### 9.4. Rear double wheels (solid wheels). Independent brakes system.

The rear wheels are an integral part of the frame and can be removed for replacement.

- **Independent brakes system.** The rear wheels are equipped with two independent brakes located behind each wheel. To apply each brake press both pedals down (Photo 9.4.1). To unblock the brakes, pull both pedals up (Photo 9.4.2).



Photo 9.4.1

*Wheel brake on*



Photo 9.4.2

*Wheel brake off*



Photo 9.5.

*Amortisation system*

### 9.5. Amortisation system for all four double wheels

All four wheels of the Pegaz stroller are equipped with an amortisation system. It works automatically and should not be modified by the user (Photo 9.5).

## 9.6. Tilting foot

The Pegaz stroller can be equipped with a tilting foot included in the additional equipment. The tilting foot makes it easy to maneuver the stroller when it encounters small obstacles, e.g. stairs, curbs. To raise the front wheels, press the tilting foot pedal and pull both handles toward you.



Photo 9.6.  
Tilting foot

## 9.7. Adjustable ergonomic handles to push the stroller

The Pegaz stroller is equipped with adjustable push handles. In order to change their angle, press the buttons located at their ends and rotate to change their angle of inclination (Photo 9.7.1 and 9.7.2). After placing both handles in a suitable position, release the pressure. The handles can change position every 90 degrees.



Photo 9.7.1



Photo 9.7.2

*Handle to push – position a Handle to push – position b*

## 9.8. Footrest. Regulation and use

- **Installation.** To mount the footrest, slide it onto the mounting pins located at the bottom of the frame at several height levels. A click sounds when the fastening hooks come into place. After mounting the footrest, make sure that it is properly attached. To check this, pull the footrest vigorously up - it should stay at the same position.

- **Disassembly.** To remove the footrest, press the two release buttons placed on the sides of the footrest at the same time and lift the footrest. The wheelchair can not be used without a footrest.

- **Use of a footrest.** It is recommended to adjust the footrest plate properly. Child's feet should rest freely on it with its entire surface. Also child's thighs should lie parallel to the seat surface (Photo 9.8.1 and 9.8.2). There are straps in the footrest plate to secure the child's feet. It is recommended to put them on when using the stroller. The footrest plate is placed in a horizontal position as standard. However, during transport, storage, rehabilitation, getting on and off the Pegaz wheelchair, the footplate must be removed.



Photo 9.8.1  
*Minimal position*



Photo 9.8.2  
*Maximal position*

### 9.9. Seat depth regulation.

Pegaz stroller has the function of adjusting the seat depth. Thanks to the possibility of unfolding its front part, we can extend the seating area by 5cm. To do this, press both buttons located on the sides of the front part of the seat simultaneously and bend the adjustable element to obtain the position of the main part of the seat (Photo 9.9.1). To fold the seat, press the buttons located on the sides of the front part of the seat simultaneously and fold the adjustable element down or under the main part of the seat (Photo 9.9.2).



Photo 9.9.1

*Seat depth regulation – minimal position*



Photo 9.9.2

*Seat depth regulation – maximal position*

### 9.10. Backrest high adjustment

The Pegaz stroller has the function of adjusting the backrest height (Photo 9.10.1). Thanks to the option of installing metal extensions to the upper part of the backrest, we can extend the backrest by 10cm. To do this, detach the upper part of the backrest upholstery, insert the extension into the holes of the backrest tubes and fasten the upper part of the upholstery again (Photo 9.10.2).



Photo 9.10.1

*Backrest high regulation max / min*



Photo 9.10.2

*Extensions*



### 9.11. Backrest tilt angle adjustment

The Pegaz stroller has the function of a two-stage regulation of the backrest angle (Photo 10.11.1 and 10.11.2). To adjust it, release the strut located directly behind the backrest by lifting it up (Photo 10.11.3), move down the backrest mounts located on the sides of the frame and tilt the backrest from the basic position to the reclined position (Photo 10.11.4). In order to lock the deflected position, the strut should be locked again by lowering it down and pressing it properly (Photo 10.11.5).



Photo 10.11.1

*Backrest tilt angle regulation – basic position*



Photo 10.11.2

*Backrest tilt angle regulation – modified position*

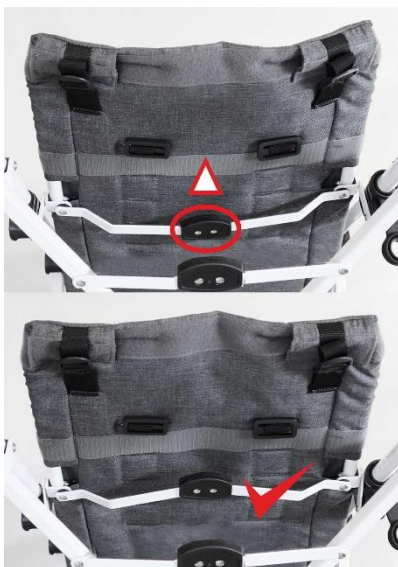


Photo 10.11.3

*Release of the spreader*

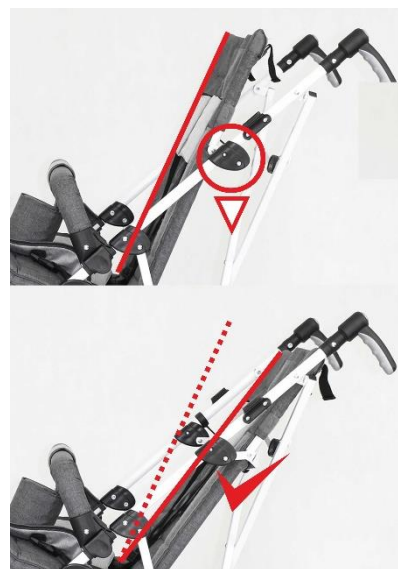


Photo 10.11.4

*Lowering mounts*



Photo 10.11.5

*Locking the spreader*

## 10. Standard equipment

### 10.1. Five points safety belts

The Pegaz stroller comes standard with professional, five-point adjustable seat belts (Photo 10.1.1).



Photo 10.1.1



Photo 10.1.2

*Safety belts adjustment – minimum    Safety belts adjustment - maximum*

- **Adjustment.** Adjust the five-point belts to your child before attaching them. To adjust the length of the straps, move the adjusters on the straps up or down and move the protectors (Photo 10.1.1 and 10.1.2) accordingly. To adjust the height of the attachment of straps to the backrest, the harness attachment should be moved from the low attachment to the high attachment or conversely.
- **Use.** To attach a five-point harness to your child, put the harness on the right and left child's arm and then attach both harnesses to the belt buckle that is fastened on the seat. To remove the five-point belts from your child, press the button, on the buckle and remove both braces from the buckle, then slowly remove these braces from the right and left child's arm. Each time you change the angle of the backrest, you need to readjust the belts length.

### 10.2. Transport lock

The stroller is equipped with a transport lock, which serves to protect the folded (closed) stroller frame against its automatic opening during transport and storage. (Photo 10.2). The transport lock must be attached after folding the frame and removed before unfolding the frame. Do not extend the trolley frame if the lock on the frame, as this may damage the the structure of the frame construction.



Photo 10.2

*Transport lock*



### 10.3. Safety barrier



Photo 10.3  
*Safety barrier*

As standard equipment, the Pegaz stroller has a safety barrier that is designed to prevent a child from falling out of the stroller (Photo 10.3). The safety barrier is used alternately with a therapeutic table (included in the additional accessories set) and is a structural element of the wheelchair.

- **Installation.** The element should be mounted on the stroller frame by using plastic latches, which look the same on both ends. After applying both latches at the appropriate height, press them so that they completely cover the frame.

**WARNING!** Due to structural reasons, the stroller must have a footrest and a safety barrier or therapeutic table on.

### 10.4. Shopping basket

The shopping basket is part of the standard equipment of the Pegaz stroller (Photo 10.4). It's right under the seat. Its maximum load capacity is 2kg.



Photo 10.4  
*Kosz na zakupy*

### 10.5. Wedge

The stroller is equipped with a wedge, which should be placed between the child's thighs after placing the child in the stroller. To install the wedge, attach it with Velcro on the front or back of the apron (Photo 10.5.1 and 10.5.2) and then wrap it with a band to strengthen it.



Photo 10.5.1  
*Wedge – position maximal*



Photo 10.5.2  
*Wedge – position minimal*

## 10.6. Abduction and stabilizing belts

Abduction and stabilizing belts are used for additional pelvis stabilization and prevent the child from slipping out of the stroller (Photo 10.6). The correct way to use them is to wrap the child's thighs so that his pelvis adheres as close as possible to the backrest. This helps keep a healthier and more comfortable position of the child in the stroller.

- **Adjustment.** To adjust the length of the support belt, you must move the adjusters on the front belt straps up or down.
- **Use.** To put on abduction and stabilizing belts the child should be seated on the fitted straps, then wrap the child's leg and belt buckles fasten both to the front of the two tapes. In order to remove abduction and stabilizing belts, unfasten the belt buckles of the two front straps.



Photo 10.6

Abduction and stabilizing belts

## 11. Additional equipment

### 11.1. Adjustable canopy with window.

The Pegaz wheelchair can be equipped with an adjustable canopy with window, that helps protect the child from the sun or rain (Photo 11.1.1). The elements of the booth are: frame (front and rear arch), sunroof adjustment mechanism (right and left), two snaps with red buttons fastening the roof to the frame, roof upholstery, and a window with a mosquito net with a zipper.

- **Installation.** The fastening element for the adjustable hood with the window are two plastic elements located on the outer sides of the upper part of the frame at the height of the headrest. To assemble the adjustable canopy with the window, set the backrest to the upright position, and then simultaneously slide both visor latches to the frame mounts. Mount the booth with the window in such a way that the back part of the booth's upholstery overlaps the back of the stroller.
- **Disassembly.** To disassemble the adjustable canopy with window, press the two red buttons on the canopy fastening latches, then slide the booth fasteners out of the mounts on the frame.
- **Use.** In order to change the size of the booth's window area, hold the front arch and the rear arch of the booth's frame and bring the arches closer or further to each other to set the desired position. The mechanism adjusting the size of the booth's surface locks automatically. The booth has a removable window with a mosquito net, ensuring free air circulation. The window opens and closes with a zipper.



Photo 11.1.1

Adjustable canopy with window

## 11.2. Therapeutic table



Photo 11.2

*Therapeutic table*

The Pegaz stroller can be equipped with a non-slip therapeutic table with a maximum load of 3 kg (Photo 11.2). To assemble the table, slide the open fastening clamps onto the main element of the frame, and then tighten the clamps until they are completely closed. Make sure it is securely attached by pulling it vigorously - the table should remain in place. To disassemble the table, unfasten the clamping clamps and remove the table from the trolley frame.

The therapy table is used alternately with a safety barrier and is an essential structural element of the wheelchair.

**WARNING!** For structural reasons, the stroller must have a footrest and a railing or therapeutic table.

## 11.3. Loin cushion

The loin cushion is an additional piece of equipment that increases the comfort of using the wheelchair by softening the backrest in the lumbar section (Photo 11.3). The cushion is installed by attaching it to the back with velcro.

## 11.4. Headrest

The headrest is an element of optional equipment (Photo 11.4). It performs both softening and stabilizing functions. To adjust the height of the headrest, attach it with velcro at the correct height of the backrest. Then, for better stability, drag the headrest straps fitted with velcro through the backrest openings and attach them behind the backrest.

## 11.5. Narrowing pads

The Pegaz stroller is equipped with two narrowing inserts. They are used to adjust the inside dimensions of the seat width (Photo 11.5). The inserts are mounted by attaching them with the longest vertical edge of the pad to the outer edges of the backrest and to the side elements of the trolley frame using a velcro strap



Photo 11.3

*Loin cushion*

Photo 11.4

*Headrest*

Photo 11.5

*Narrowing pads*

### 11.6. Tourist bag

The large and elegant travel bag is mounted on two handles, located on the inside of the handle to push the trolley (Photo 11.6). The maximum bag load is 2kg.

**WARNING!** Exceeding the maximum bag load has a negative effect on the stability of the stroller and there is a danger for the child and other people!



Photo 11.6  
*Tourist bag*

### 11.7. Raincover

The Pegaz stroller can be equipped with a raincover that helps protect the child from moisture (Photo 11.7). This optional equipment requires to be installed to other optional equipment - an adjustable canopy with window.

**Assembly.** To install the raincover, put it on top of the entire adjustable canopy with window, and then cover the seat and footrest. After making sure that the foil is applied properly, attach it to the frame and footrest - depending on the location of the fastener.

### 11.8. Mosquito net

The Pegaz stroller can be equipped with a mosquito net, which prevents insects from coming into direct contact with the child (Photo 11.8). This accessory requires to be installed to another accessory - an adjustable canopy with window.

**Assembly.** To install the mosquito net, put it on top of the entire adjustable hood with window, and then cover the trolley seat and footrest with a net. After making sure that the net is applied properly, attach it to the frame and footrest - depending on the location of the fastener.

### 11.9. Sleeping bag

The Pegaz stroller can be equipped with a sleeping bag for legs, increasing the child's comfort during colder days (Photo 11.9). This piece of equipment requires to be installed to the safety barrier.

**Assembly.** In order to mount the sleeping bag, put it on the footrest from below, and attach it from above through the safety barrier. Then fasten it with snaps to the frame of the stroller.

### 11.10. Sunshade

The Pegaz stroller can be equipped with an umbrella that helps protect the child from the sun (Photo 11.10).

**Assembly.** To install the umbrella to the trolley, loosen the clamp by unscrewing the thread located on the side, then slide it on, ensuring that the rubber washer remains in place. After choosing the desired height, tighten the thread until the clamp on the trolley frame is fixed properly.





Photo 11.7  
*Raincover*



Photo 11.8  
*Mosquito net*



Photo 11.9  
*Sleeping bag*



Photo 11.10  
*Sunshade*

## 12. Technical specification of Pegaz stroller

<b>Manufacturer</b>	MyWam Kupiec, Bartold, Angres spółka jawna
<b>Address</b>	ul. Szczecińska 10, 41-516 Chorzów, Polska
<b>Type</b>	Special children's stroller
<b>Model</b>	Pegaz

The total length of the stroller	103 cm
Total width	56 cm
Total height	130 cm
Total weight	od 10 kg
Seat depth	25*/30 cm
Seat width	30*/38 cm
Seat height	50/60 cm
Backrest width	37 cm
Distance between footrest step and seat	20/29/40 cm
Folded stroller length for transport or storage	124 cm
Folded stroller width for transport or storage	28 cm
Folded stroller high for transport or storage	39 cm
Maximum basket load capacity	2 kg
Maximum carrying capacity of the touristic bag	2 kg
Maximum truck load (user weight + additional load weight)	40 kg

The Manufacturer reserves the right to make changes to technical parameters.  
Measurement tolerance +/- 20mm, +/- 1,5kg, +/- 1°.

### 12.1. User parameters

Maximum child height	80 – 140 cm
Maximum child's weight	40 kg

### 12.2. Material composition of the stroller

Powder coated steel and aluminium, fabrics 100% polyester and plastic (PP, PCV, ABS, PA, PE, PU).

## 13. Delivery and tools

The Pegaz wheelchair is delivered in a cardboard box and pre-assembled. The original packaging should contain the following items:

- Folded stroller frame with integrated seat, backrest and wheels, footrest, safety barrier and wedge \* or abduction and stabilizing straps \* (according to the order)
- Additional equipment (according to the order)
- User manual with warranty card

Before use, make sure that the package contains all the elements and none of them has been damaged, e.g. during transport. If it is found that the received package is missing or damaged, please



contact the Seller and the Manufacturer in order to receive the missing elements or replace the product with a new, fully complete and functional one.

The multi-functionality and very rich equipment of the Pegaz wheelchair allows you to adjust the stroller to your personal requirements. To do this, read the entire instructions carefully before use.

No tools are required to assemble and disassemble the Pegaz stroller.

#### 14. Assembly and disassembly of the stroller

**Stroller assembly.** The product pre-packed in the original box is already pre-assembled. Complete the installation by completing the following steps:

- unfold the stroller frame by adjusting the parameters: lower and stiffen the spreaders at the height of the backrest in the rear of the stroller and at the height of the shopping basket in the front of the stroller.
- activate / deactivate the front direction lock
- adjust the height and angle of the handle to push the trolley
- set the angle of the backrest
- adjust the length of the five-point belts
- put on a footrest at the desired height
- put on a safety barrier or a therapy table
- install appropriate equipment and adjust parameters, for example:
  - put narrowing inserts
  - remove / install the extension on the backrest
  - set the seat depth
  - adjust the abduction and stabilizing belts
  - adjust the canopy with the window
  - install and adjust the wedge
  - set up a shopping basket
  - put on selected items of equipment dedicated to the seasons:
    - sleeping bag for the legs
    - mosquito net
    - raincover
    - umbrella
- **Dismantling the stroller**

To disassemble the stroller:

  - a) Remove accessories
  - b) Remove the footrest
  - c) Remove the safety barrier / therapy table
  - d) Block the front wheels
  - e) Apply the brakes to the rear wheels
  - f) Raise the struts at the height of the backrest in the rear of the stroller and at the height of the shopping basket in the front of the stroller
  - g) Fold in the truck, closing it with the transport lock

## 15. Preparation for use

Before using the truck:

- Make sure that the child's footwear is comfortable and stabilizes the ankle well.
- Make sure that the child is not placed in the stroller without clothing covering the body in places of contact with the upholstery of the stroller. This is to prevent chafing of the child's body and hygiene of the child.
- For children who have trouble holding urine, put a special oilcloth on the stroller seat, which can be purchased at pharmacies or medical stores.
- When using any additional aids such as venflons, drains, bags, catheters, etc., make sure that they are well secured against falling out, pulling out, clamping and do not interfere with the use of the trolley.

**In order to start using the truck properly, all the following tasks must be performed:**

- Check the technical condition of the stroller and all its components. If the technical condition of the stroller or its individual component is bad or the user has any doubts about it, don't use the stroller and contact the Seller and the Manufacturer.
- Place the stroller on a flat and secure surface.
- Unfold the truck correctly
- Apply both brakes.
- Make all necessary adjustments to the truck.
- Prepare five-point belts, straps on the footrest and dehydrating-stabilizing straps, if these accessories are available.
- Place the child in the stroller and make sure that all the adjustments previously made are appropriate and safe for the child.
- Attach the child to five-point belts and straps on the footrest.
- Put on the child other necessary wheelchair equipment, if available.

## 16. Using the stroller

### 16.1. Carrying the stroller

Appropriate way to carry a stroller is to drive it using its wheels. In cases where this is not possible (e.g. to transport the trolley in a car), for safety reasons, the stroller can be lifted and moved with both hands after dismantling the additional equipment to reduce the carrying weight.

Be careful when moving or lifting the stroller's frame. The frame should be moved when the transport lock is folded and attached, holding it by a fixed element of the frame.

For safety reasons, do not lift the unfolded stroller with accessories, as this may damage the stroller.

Do not lift the stroller by its footrest or safety barrier, therapy table or any other equipment, as this may damage the product and endanger the safety of others.

### 16.2. Getting on and off the stroller

During getting on and off the stroller, place it as close as possible to a stable and safe place where the user will change. After making sure that the rear brakes are applied and the footrest has been removed to avoid standing on it, the user can be placed in the wheelchair.

### 16.3. Correct position in a stroller

For safety and health reasons, it is recommended to place user's pelvis and back as close as possible to the backrest to support the spine properly (optimal support of all parts of the spine so that there is no free space).

Thighs should be supported on the largest surface, along their entire length, which is used to adjust the depth of the seat of the stroller and the feet should rest on the footrest plate, which prevents slipping, maintains optimal tension of the leg muscles and carries part of the weight from higher parts of the body.

### 16.4. Riding in a stroller

When using the stroller, always hold it by the handles to push to prevent the stroller from rolling. Release the rear brakes while holding the push handle and push the stroller in the desired direction.

### 16.5. Moving on slopes

Always position the stroller forwards when climbing or driveways. Limit the speed on sloping surfaces and move as slowly as possible. The stroller assistant (parent, guardian) must have sufficient strength to control the stability of the stroller.

### 16.6. Overcoming thresholds or curbs

When overcoming small obstacles (small thresholds, small curbs, etc.), block the tilting foot with the foot and the handle to push the stroller downwards and towards you to raise the front wheels of the stroller and carefully drive the front wheels over the obstacle. Then slowly pull the push handle upwards away from the ground to raise the rear wheels of the stroller and carefully drive the rear wheels onto the obstacle.

### 16.7. Transport in a car and other vehicles

To transport the Pegaz stroller in the car, remove all accessories from the stroller and store them in a safe place. Then dismantle the stroller and place it in the trunk. If the dismantled stroller is not in the trunk of the car separated from the passenger compartment, then the stroller frame must be securely attached to the vehicle. For this purpose, unused seat belts may be used.

- In order to transport a stroller in other vehicles, such as trains, trams, buses, planes, ships, etc., please ask the persons responsible for transport in these means of transport about the conditions of safe travel to the destination with a stroller and the appropriate requirements.
- In order to transport the stroller on other vehicles, remove all accessories from the stroller and store them in a safe place. Then dismantle the stroller and place it in the baggage storage area of the given vehicle. If the dismantled stroller is not in the trunk of the vehicle separated from the passenger compartment, the stroller frame must be securely attached to the vehicle.

The Pegaz stroller should not be used as a seat to transport a child in a vehicle.

## 16.8. Cleaning and care

Small leaks and stains can be removed from the surface with a kitchen sponge and using clean, warm water. Stains should be removed immediately after they appear on the surface of the stroller. Do not rub - rubbing will spread the stain to a larger surface and may distort the texture of the fabric. The treatment of water stains such as stains from food, drinks and vegetable oils is as follows:

- Dry the stain by pressing a white paper towel, handkerchief or absorbent clean cloth over its entire surface.
- Soak a small piece of cloth with a weak solution of detergent and warm water (or cold water in the event of blood staining) and rinse the stain by removing it from the outside to the inside to prevent it from growing. Do not use a too wet cloth, as excessive amount of liquid may cause ripples after drying or enlargement of the stain and damage of the upholstery or foam inside.
- It may be necessary to repeat the above operation depending on the type of stain and the amount of stained material. The above operation should be continued until the stain is completely removed, but always remember to remove excess moisture from the stained material before rinsing again.
- Then rinse the material using only lukewarm water without detergent, using a clean sponge or a clean white cloth. As above, rinse the material in place after the stain in the right direction, i.e. from the outside in.

Disinfectants that do not contain organic solvents can be used in accordance with the manufacturer's instructions. Do not dip the fabric. Make sure the product is always dry before use. Do not use organic solvents or dry cleaning agents.

For disinfection, you can use a small amount of ordinary household bleach, diluted with water with a maximum concentration of 3% bleach.

**ATTENTION!** The manufacturer is not liable for damages resulting from the use of inappropriate cleaning products.

## 17. Storage of the stroller

The stroller should be stored in a clean, dry and well ventilated room, preferably at 0-35 Celsius degrees.

### 17.1. Preparation of the stroller for long-term storage

In order to prepare the Pegaz stroller for long-term storage, secure the product and perform all service operations in the "Periodic Inspections" chapter as if they had occurred.

The stroller should be stored in a clean, dry and well ventilated room.

### 17.2. Corrosion protection

To avoid corrosive damage to the stroller, it should not be used or left in wet and damp rooms. When the truck is wet, wipe all wet parts with a soft cloth and dry with a light oil preservative. This does not apply to the upholstery of the stroller, the provision of cleanliness and maintenance of which is described in detail in the Cleaning and care chapter.

As a standard, the stroller is covered with the highest quality coatings. In the event of damage to the varnish, protect the damage by applying a touch-up compound using the varnish proposed by the varnish Manufacturer.

Regular cleaning and lubrication of the stroller mechanisms with light oil guarantees long and trouble-free use of the stroller.

### **18. Re-use**

The product is suitable for reuse.

All reusable products are subject to a particular burden.

Therefore, their features and performance may deviate from the new product to such an extent that the safety of patients and third parties during their use is at risk.

The product with proper care and maintenance retains its quality for a period of 3 years and this is the expected period of its use.

For secondary use, please:

- Remove all upholstered parts and replace them with new ones.
- The stroller frame with wheels and steering elements should be thoroughly cleaned and disinfected with commercially available cleaning and disinfecting agents.
- The product must then be checked by an Authorized Service Center (see Chapter 24) or the Manufacturer for wear and damage.
- All worn parts and components that do not match or are unsuitable for the user should be replaced with new ones.
- The product prepared in this way can be put back into use.

### **19. Service and periodic inspections**

For safe and trouble-free use of the Pegaz stroller, it must be regularly and thoroughly inspected and serviced as indicated in the table below.

In a situation where the stroller user is not able to carry out the activities himself, he should return the stroller to a specialized stroller service point or directly to the Manufacturer, who will properly review and will perform necessary service activities.

These activities may be charged extra.

All repairs should be carried out in specialized, authorized Services (see point 24) or directly at the Manufacturer.

Only original spare parts that are supplied or recommended by the Manufacturer may be used in the stroller. Producenta.

#### **Service should be performed:**

##### **Each time before using the stroller**

- Check the efficiency of the central brake - After activating the central brake, both rear wheels of the stroller should be locked and cannot rotate.
- Check the attachment of structural elements - It is necessary to check all mechanisms of the stroller and its movable elements, in particular, pay attention to: the correct functioning of the fivepoint seat belts, properly mounted safety barrier or therapeutic table, properly locked backrest angle adjustment mechanism and footrest correct mounting. o
- Check the condition of the tires - Check that the tires on all four wheels are not damaged or worn. o Inspection of solid frame parts, seat with backrest and footrest frame - Visually inspect the condition of the structural profiles. These elements must not be damaged or cracked.

### Every one month

- Cleaning the stroller - Clean any moving parts, bearings, adjustment mechanisms with a soft cloth and then lubricate them with a few drops of light oil. Be careful not to soil the upholstery of the stroller or your own clothes.
- Check all bolted connections for looseness - Carefully check that all bolts and knobs in the stroller are properly tightened.

### Every six months:

- Sprawdzenie o Check the cleanliness of the upholstery and inspect the general condition of the stroller - Do not allow the stroller to be neglected as described in the chapter „Cleaning and care”.

### 20. Disposal

Disposal and recycling: the used stroller should be returned to a recycling center in accordance with the regulations in force in the given country.

### 21. Compliance with requirements for medical devices

The Pegaz stroller meets the essential requirements of Council of Europe Directive 93/42 EEC amended by 2007/47/EC for medical devices.

The Pegaz stroller, in accordance with Directive 93/42 EEC, is a non-invasive, inactive medical device of class I according to rule 1.





MYWAM Kupiec, Bartold, Angres spółka jawna  
ul. Szczecińska 10, 41-516 Chorzów, Polska  
tel. 0048 32 733 11 31  
[mywam@mywam.pl](mailto:mywam@mywam.pl)



## DEKLARACJA ZGODNOŚCI WE

### EC DECLARATION OF CONFORMITY

<b>Wytwórca:</b> <i>Manufacturer:</i>	<b>MYWAM Kupiec, Bartold, Angres spółka jawna</b> ul. Szczecińska 10, 41-516 Chorzów, Poland
<b>Produkt:</b> <i>Product:</i>	<b>Wózek inwalidzki specjalny MEWA</b> Specialist Stroller MEWA
<b>Produkt:</b> <i>Product:</i>	<b>Wózek inwalidzki specjalny PEGAZ</b> Specialist Stroller PEGAZ

*Deklaruję z pełną odpowiedzialnością, że wózek inwalidzki specjalny Mewa oraz Pegaz, będące wyrobem medycznym klasy I, są zgodne z wymaganiami dyrektywy Rady Europy o wyrobach medycznych - 93/42/EEC z 14 czerwca 1993 roku zmienionej przez 2007/47/EEC z 21 marca 2010 roku, przyjętej w Polsce na mocy ustawy o wyrobach medycznych z dnia 20 maja 2010 r. (Dz. U. nr 107, poz. 679), oraz zgodne z normami zharmonizowanymi:*

*I solely declare that the specialist Stroller Mewa and Pegaz are a class I device, and there are in conformity with the requirements of EU Council Directive for Medical Devices 93/42/EEC of 14 June 1993 amended by 2007/47/EEC of 21 March 2010, implemented in Poland under the Medical Devices Act of 20 May 2010 (Dz. U. nr 107, poz. 679), and are in conformity with harmonized standards:*

PN-EN 12183:2014	PN-EN 12182:2012	PN-ISO 7176 part 8	ISO 7176 part 1,3,5,7
PN-EN 1021-1:2007	PN-EN 980:2010	PN-EN 1041:2010	PN-EN ISO 14971:2011



Chorzów, 29 marzec 2018 r. / 29th of March 2018

.....  
Data/Date



Wojciech Bartold

*Wojciech Bartold*  
-wspólnik-

Podpis/Signature








**MYWAM**  
Kupiec, Bartold, Angres Spółka Jawna  
41-516 Chorzów, ul. Szczecińska 10  
[www.MyWam.pl](http://www.MyWam.pl)

tel. 32/733 11 31, mail: [mywam@mywam.pl](mailto:mywam@mywam.pl)  
NIP: 627 273 67 93, Regon: 24348050  
KRS: 0006487471

## 22. Notes

### 23. List of authorized service points

POLAND  
MyWam Kupiec, Bartold, Angres Sp. Jawna  
ul. Szczecińska 10, 41-516 Chorzów, Polska  
tel.: +48 32 733 11 31  
website: [www.mywam.eu](http://www.mywam.eu)  
e-mail: [mywam@mywam.pl](mailto:mywam@mywam.pl)

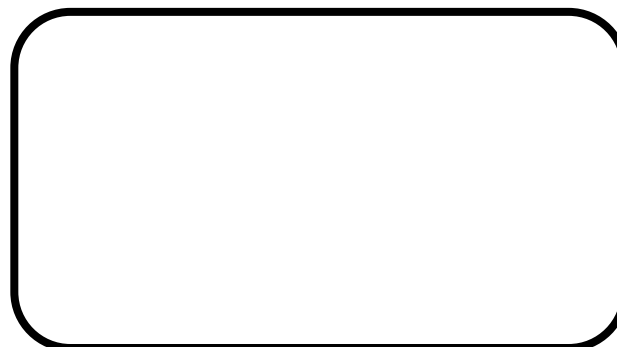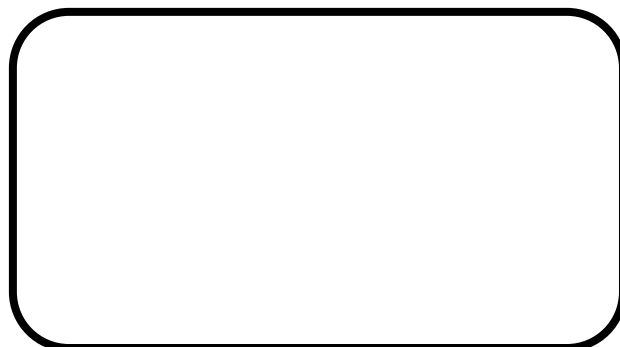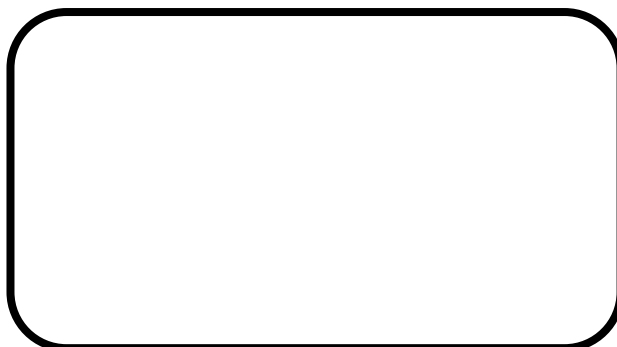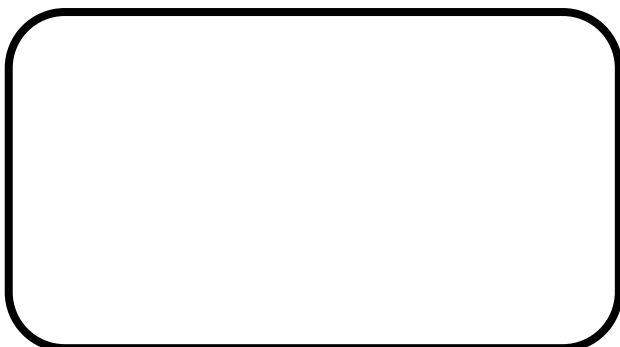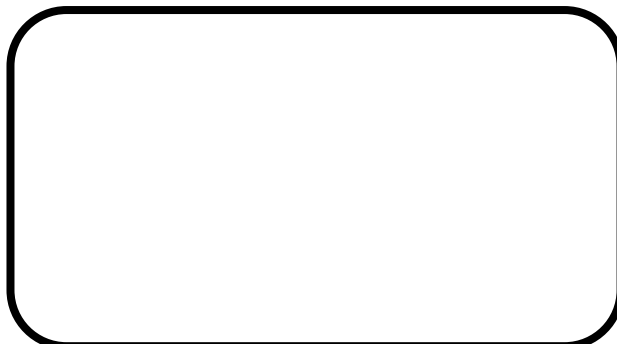

## 24. Warranty Card

6. Warranty Card Series and number: .....

Date of purchase: ..... - ..... - .....

Seller stamp and signature:

.....  
.....  
.....

### Warranty conditions

The manufacturer grants the buyer a warranty for the purchased stroller, for a period of 24 months from the date of purchase.

Defects revealed during this period will be removed free of charge.

The Buyer submitting a complaint on the stroller should deliver the stroller to the Seller or to the Manufacturer in the original factory packaging, together with the elements of the stroller protected against damage.

Damage caused by insufficient securing of the stroller shipment and its components is not subject to warranty repairs.

The complainant may not demand gratification if he improperly marks or incorrectly insures the shipment of the stroller and its components.

The cost of shipping the complained stroller to the Seller or to the Manufacturer shall be borne by the buyer complaining the stroller.

The warranty period is extended by the time needed to repair the stroller.

The activities provided for in the manuals, which the user is obliged to perform on their own and at their own expense, cannot be performed as part of a warranty repair.

The buyer has the right to replace the complained stroller with a new stroller if the same damages or defects that have already been removed twice and which prevent the stroller from being used are still present.

The stroller should be used in accordance with these operating manual.

Unauthorized use of the stroller, inconsistent with these operating manual or inconsistent with the principles of correct operation will void the warranty.

Making any structural changes to the stroller will void the warranty.

Damage to the stroller caused by: improper operation, improper maintenance, incorrect storage of the stroller, mechanical damage as well as natural abrasion and wear of parts (e.g. upholstery, wheels) are not covered by the warranty. They are repaired at the user's expense.

The manufacturer warns against performing warranty repairs at unauthorized service facilities. Finding such repair or unauthorized alteration of the stroller will void the warranty.

The warranty card is valid provided that it has: the date of sale entered, confirmed by the Seller's company seal, the Seller's signature, type and serial number of the stroller.

The warranty card is valid as long as it does not contain deletions and corrections.

The stroller manufacturer runs its paid post-warranty service.

In matters not covered by this warranty card, the provisions of the Civil Code shall apply.

All matters related to the warranty