



**POLISH MANUFACTURER
OF REHABILITATION EQUIPMENT
FOR CHILDREN**

MANUFACTURER / ADVERTISING ENTITY:
MyWam Merchant Bartold Angres sp.j.

delfi

bath chair

The DELFI bath chair is designed for young patients up to 30 kg body weight. Its basic task is to enable the guardian to perform bathing and hygienic activities freely on the user placed on it, with proper protection and in an optimal position for him. The relatively large surface of the bath chair allows the guardian to make free movements. The DELFI bath chair is intended for use in the bathtub or shower.

A number of necessary protections, such as an adjustable stabilizing headrest, torso stabilizing belt and leg stabilizing belts, significantly affect user safety during bathing. The bath chair supports are wrapped in foam, ensuring adhesion and protecting the surface from scratches.

Adjustable individual segments of the bath chair – headrest angle, the angle of the backrest and footrest in combination with the possibility of adjusting the height of the backrest and the depth of the seat allows you to optimize the position of the patient and adapt the product to individual user parameters in the indicated range.

INDICATIONS

Clinical indications include cerebral palsy, meningitis – spinal hernia, dystrophy, paresis, limb damage, etc.

AGAINST / RISK

Incorrect positioning and long-term keeping of a person in one position can have negative effects.

IMPORTANT

The user can use the bath chair only with the help of a guardian. Use of the product other than those described in the instructions for use is prohibited.

The device should be used as directed by the attending physician or physiotherapist.



illustrative photos

CONSTRUCTION

The DELFI bath chair has a lightweight, aluminum frame that is a skeleton with movable segments adjustable at neck, hips and thighs. It also has regulation the height of the backrest and the depth of the seat, which allows the device to be adapted to the individual needs of the patient.

The light frame is made of aluminum tubes filled with polyurethane foam, preventing water from entering them. The perforated upholstery made of synthetic fabric pulled on it is indifferent to the action of water, and its structure affects the short drying time of the bath chair.

MODIFICATION OF PARAMETERS

The DELFI bath chair has a specific range of seat depth and backrest height adjustment. In combination with flexible upholstery, we gain a device tailored to the individual needs of a small patient.




CE MD

**This is a medical device.
Use it according to the instructions for use or label.**

bath chair functionality


HEADREST RECLINE ANGLE

The headrest angle adjustment function helps in adjusting the safe and comfortable position of the child's head during bathing, so that water does not fall into the child's eyes, e.g. when washing hair.

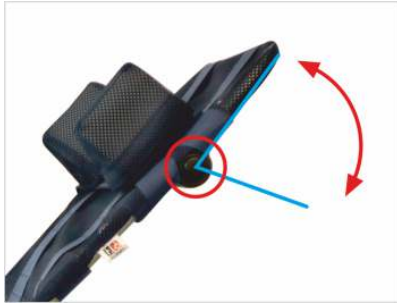


BACKREST RECLINE ANGLE

Adjusting the back angle allows the user to be seated in the optimal position, which guarantees comfort and safety during bathing.

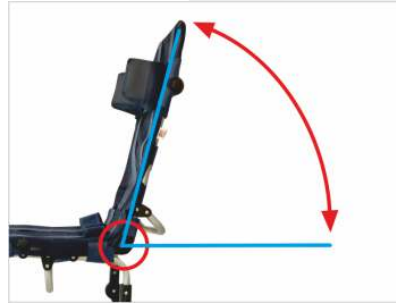

FOOTREST RECLINE ANGLE

Adjusting the angle of the leg backrest allows you to adjust the optimal position of the child's legs during bathing, enabling the caregiver to freely perform hygiene activities in the most convenient position.

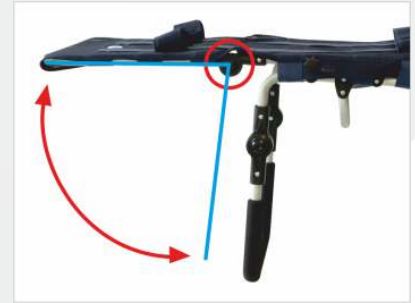

necessary accessories

HEADREST

The adjustable headrest protects and stabilizes the child's head during bathing, preventing it from falling uncontrollably.




STABILIZING BELT

The elastic torso stabilizing belt stabilizes the user during bathing, preventing the child from sliding off the bath chair.



STABILIZING STRAPS

Two individual leg stabilizing straps allow you to protect and immobilize the user's legs during hygienic activities, and if necessary for quick unfastening.

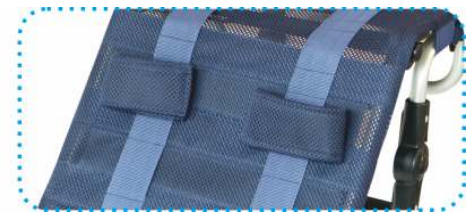

illustrative photos

PARAMETERS

LENGTH OF THE BATH CHAIR	136,8 - 148,8 cm
WIDTH OF THE BATH CHAIR	41,8 cm
HEIGHT OF THE BATH CHAIR	98 cm
TOTAL WEIGHT OF THE BATH CHAIR	4,9 kg
SEAT DEPTH	38 - 42 cm
SEAT WIDTH	40 cm
BACK HEIGHT	60,5 - 68,5 cm
WIDTH OF THE BACKREST	40 cm
LENGTH OF THE FOOTREST	38 cm
SEAT ANGLE	4,5 °
BACKREST ANGLE	10 - 65 °
FOOTREST ANGLE	+2 - 52 °
ANGLE BETWEEN THE BACKREST AND THE SEAT	105 - 166 °
MAXIMUM PATIENT WEIGHT	30 kg



delfi

www.mywam.eu



MyWam Kupiec Bartold Angres sp.j.
41-516 Chorzów, ul Szczecińska 10
NIP: 627 273 67 93
e-mail: mywam@mywam.pl

NOTE: The information contained in this leaflet is for illustrative and educational purposes only, therefore: 1) does not constitute a commercial offer under commercial law, 2) cannot be used as a basis for making decisions about taking or not taking treatment or other medical procedures. The interpretation should be made by a specialist doctor. The manufacturer reserves the right to make changes to the products.