

The **DELFI bath seat** is designed for young patients weighing up to **30 kg**. Its primary task is to allow the caregiver to freely perform bathing and hygiene activities on the user placed on it, with proper protection and in an optimal position for the user. The relatively large surface area of the chair allows the caregiver to perform free movements. The **DELFI bath seat** is designed for use in the bath or shower.

A number of necessary safety features, such as an adjustable stabilizing headrest, a torso stabilizing belt and leg stabilizing straps significantly contribute to the user's safety while bathing. The recliner's supports are wrapped in foam, providing grip and protecting surfaces from scratches.

Possibility of adjusting individual segments of the bath seat – the angle of the headrest, the angle of the backrest and the footrest, combined with the possibility of adjusting the height of the backrest and the depth of the seat, allows you to optimize the patient's position and adapt the product to the individual user parameters within the indicated range.

INDICATIONS

clinical indications include cerebral palsy, meningitis, spinal hernia, dystrophy, paresis, limb damage, etc.

CONTRAINDICATIONS

Injuries to the upper limbs that disenable the user to support himself on the walker.

delfi
BATH SEAT



CE MD MEDICAL DEVICE - CLASS I

NEW

CONSTRUCTION

The **DELFI bath seat** has an aluminum frame construction with movable segments adjustable at neck (1), hip (2) and thigh (3) heights. It also has adjustable backrest height and seat depth, allowing the device to be adjusted to the individual patient's needs.

The lightweight frame is made of aluminum tubes filled with polyurethane foam, preventing water from entering them. Stretched over it, the perforated upholstery made of synthetic fabric is inert to water, and its structure affects the short drying time of the deckchair.

This lightweight (4.9 kg) and compact piece of equipment works well at home in both the bathtub and shower stall.



MODIFICATION OF PARAMETERS

The **DELFI bath seat** has a specific adjustment range for seat depth and backrest height. Combined with flexible upholstery, we gain a device tailored to the individual needs of a small patient.


THIS IS A MEDICAL DEVICE.

USE IT IN ACCORDANCE WITH THE INSTRUCTIONS FOR USE OR THE LABEL.

BATH SEAT FUNCTIONALITY

HEADREST RECLINE ANGLE

The headrest angle adjustment function helps in adjusting the safe and comfortable position of the child's head during bathing, so that water does not fall into the child's eyes, e.g. when washing hair.

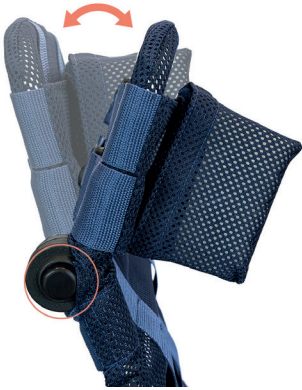

BACKREST RECLINE ANGLE

Adjusting the back angle allows the user to be seated in the optimal position, which guarantees comfort and safety during bathing.



FOOTREST RECLINE ANGLE

Adjusting the angle of the leg backrest allows you to adjust the optimal position of the child's legs during bathing, enabling the caregiver to freely perform hygiene activities in the most convenient position.



NECESSARY ACCESSORIES

HEADREST

The adjustable headrest protects and stabilizes the child's head during bathing, preventing it from falling uncontrollably.



STABILIZING BELT

The elastic torso belt stabilizes the user during bathing, preventing the child from sliding off the bath seat.



STABILIZING STRAPS

Two individual leg stabilizing straps allow to protect and immobilize the user's legs during hygienic activities, and if necessary for quick unfastening.



TECHNICAL PARAMETERS OF THE BATH SEAT

seat width: 40 cm
 effective seat depth: 38 – 42 cm
 backrest height: 60,5 – 68,5 cm
 backrest width: 40 cm
 length of the footrest: 38 cm

total weight: 4,9 kg
 bath seat length: 136,8 – 148 cm
 bath seat height: 98 cm
 bath seat width: 41,8 cm

IMPORTANT NOTE! The choice of a medical device appropriate for a specific disease entity and condition of the patient must be made with the help of a specialist doctor or physiotherapist. Before use, read the operating manual and the label or consult your doctor, as this medical device may not be suitable for you.