

Alfa flex

SPECIAL NEEDS STROLLER FOR CHILDREN

NEW

GENERAL INFORMATION

ALFA FLEX – a special needs stroller with **a three-piece molded seat from Stabilo and „I grow with you” system**. It is dedicated to children at a height of **100 – 150 cm** and a maximum weight of **55 kg**.

This innovative stroller has been designed with the thought of maximum comfort and individual adjustment to the user's needs. Both the seat and backrest of the stroller consist of **removable Stabilo orthopedic cushions**, which provide excellent body support and stabilize the user's posture.

Combined with the possibility of **of quick and easy backrest recline, tiled seat, adjustable footrest and vacuum fixated cushions**, we obtain a comprehensive product that ensures maximum safety and functionality for many years.



CE MD MEDICAL DEVICE - CLASS I

THIS IS MEDICAL DEVICE.

USE IT IN ACCORDANCE WITH THE INSTRUCTIONS FOR USE OR THE LABEL.



RECLINING BACKREST



TILT IN SPACE SEAT



ADJUSTABLE FOOTREST

INDICATIONS

- paralysis and paresis of lower limbs various origins
- paralysis and paresis of the trunk
- weakening of postural muscles responsible for maintaining the correct body position
- cerebral palsy
- myelomeningocele
- muscular dystrophies

CONTRAINDICATIONS

In case of any contraindications to sitting position &/or passive verticalization stroller adjustments should be consulted with physiotherapist or physician-in-charge.

CONSTRUCTION

The construction of the special **ALFA FLEX** stroller consists of two basic elements: a frame with a seat and a backrest. The backrest is fixed to the seat with the help of a simple locking mechanism (check the manual for details).

Additionally, the set includes 3 types of modlable cushions – **a Stabilo headrest cushion (1)**, **a Stabilo backrest cushion stabilizing the back (2)** and **a Stabilo seat cushion with a wedge and side supports (3)**. The special construction of the cushions supports the correct positioning of the spine and reduces pressure on sensitive body points, preventing pressure sores and discomfort.



ATTENTION! Before placing your child in the stroller, make sure that the backrest locking mechanism is properly locked on both sides of the stroller; to do so grab the backrest and gently shake it. A properly installed backrest will remain in its position.

PARAMETER MODIFICATION

Stabilo vacUum posture cushions allow for most precise fitting. Adjustment is simple, allowing precise fit of the shape of the seat and backrest, which increases comfort and improves the ergonomics of the seat.

Alfa flex allows you to adjust the height of the headrest (1), height of the backrest (2) as well as seat depth (3) and footrest (4) so as to meet individual needs of the child.

1.



HEIGHT ADJUSTABLE HEADREST

2a.



2b.



BACKREST HEIGHT ADJUSTMENT

3a.




3b.




SEAT DEPTH ADJUSTMENT

4a.

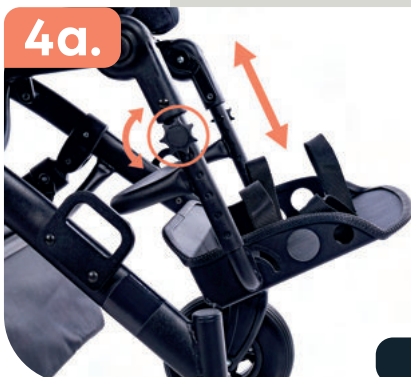

4b.

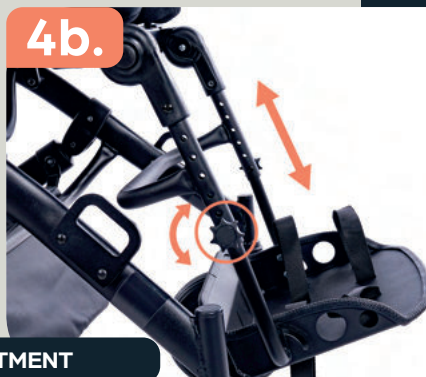

FOOTREST ADJUSTMENT

EQUIPMENT



SAFETY VEST



ADJUSTABLE PADS



SAFETY BARRIER



ABDUCTION BELT



SHOPPING BASKET



MAGNETIC WINTER GLOVES



THERAPY TRAY



CANOPY WITH WINDOW



BACKPACK



SUN UMBRELLA



STABILO HEADREST CUSHION



STABILO BACKREST CUSHION



STABILO SEAT CUSHION WITH WEDGE



RAIN COVER



SLEEPING BAG



MOSQUITO NET

TECHNICAL PARAMETERS OF THE STROLLER

seat width: 20 – 33 cm*

effective seat depth: 30 – 37 cm*

backrest height: 63 – 72 cm*

backrest width: 20 – 33 cm**

distance between footplate and seat: 33 – 43 cm, 6 positions

* values given using Stabilo cushions

** values are given with pads

total weight: 26 kg

stroller length: 100 – 145 cm

stroller height: 94 – 147 cm

stroller width: 66 cm

IMPORTANT NOTE! The choice of a medical device appropriate for a specific disease entity and condition of the patient must be made with the help of a specialist doctor or physiotherapist. Before use, read the operating manual and the label or consult your doctor, as this medical device may not be suitable for you.