



ILLUSTRATIVE PHOTO



alfa

INSTRUCTIONS FOR USE

Edition 4 / 2025-11-07

Special needs stroller ALFA

THIS IS MEDICAL DEVICE.

USE IT ACCORDING TO THE INSTRUCTIONS FOR USE OR LABEL.

PURPOSE OF THE INSTRUCTON

Instructions for use is designed for people who directly care of users with disabilities (assistants, parents, guardians), which will be transported in a stroller. It is also intended for doctors and physiotherapists operating the stroller.

ATTENTION! Before using the stroller, each User is obliged to read the instructions for use and act in accordance with its guidelines.

ATTENTION! In the event of any medical incidents related to the presence of the ALFA stroller, the Manufacturer must be inform the Manufacturer and the competent authority of the Member State in which you or the patient are domiciled immediately in writing by registered letter or e-mail with acknowledgment of receipt.

Manufacturer's contact details:

- postal address: MyWam Sp. z o.o., Plebiscytowa 3, 41-100 Siemianowice Śląskie, Poland
- contact phone number: +48 32 733 11 31
- e-mail address: mywam@mywam.eu



Instructions for use version: **EN, Edition 4 / 2025-11-07**

The instructions for use was translated using Microsoft Translator.

MyWam Sp. z o.o. reserves the right to make technical and commercial changes to the contents of the Instructions for use without prior notice. Each change will be determined by the date the instructions were last updated. The electronic version of the instructions for use is be available at www.mywam.eu

ATTENTION! Pictures given in the instructions for use are for reference only and can vary from the purchased model.

MyWam Sp. z o.o. has implemented a quality management system compliant with the ISO 13485 standard in the design, production and sale of class I medical devices. System certified by LL-C (Certification).



ISO 13485
LL-C (Certification)



TABLE OF CONTENTS

1. PURPOSE AND INDICATIONS FOR USE.....	5
2. PLATES	5
3. SYMBOLS DESIGNATION	5
4. IMPORTANT INSTRUCTIONS AND SAFETY RULES	6
5. GENERAL CONSTRUCTION OF THE STROLLER	7
6. STROLLER FUNCTIONS	9
6.1. Unfolding the stroller	9
6.2. Folding the frame of the stroller	10
6.3. Transport lock. Putting on and taking off the transport lock	11
6.4. Front swivel wheels. Front wheels directional lock	12
6.5. Rear wheels. Assembly and removal of rear wheels on the axle	12
6.6. Rear wheel amortization system	13
6.7. Push handle for the stroller.....	13
6.8. Central brake	14
6.9. Tilt foot	14
6.10. Mechanism for attaching the backrest to the frame of the stroller	15
6.11. Seat depth adjustment	16
6.12. Frame adjustment for changing seat angle.....	17
6.13. Backrest height adjustment.....	18
6.14. Frame adjustment of the backrest tilt angle	18
6.15. Footrest	19
6.16. Five-point belts	20
6.17. Headrest	21
6.18. Adjustable thoracolumbar pads support.....	22
6.19. Adjustable hip pads	23
6.20. Backrest side elements	23
6.21. Abduction and stabilization belts	24
6.22. Safety barrier	24
6.23. Adjustable canopy with window	25
6.24. Cup holder	25
7. ADDITIONAL EQUIPMENT.....	26
7.1. Adjustable therapy table (put in place of safety barrier)	26
7.2. Backpack.....	26
7.3. Shopping basket	26
7.4. Winter gloves.....	27
7.5. Wedge.....	27
7.6. Rain cover (put on adjustable canopy with window)	27
7.7. Sleeping bag for legs (worn on the safety barrier)	28
7.8. Mosquito net (attached to adjustable canopy with window)	28
7.9. Sunshade umbrella.....	28
8. TECHNICAL SPECIFICATIONS OF THE ALFA STROLLER	29
8.1. Material composition of the stroller	29

9.	DELIVERY AND TOOLS	29
10.	ASSEMBLY AND DISASSEMBLY OF THE STROLLER	30
11.	PREPARATION FOR USE.....	30
12.	MOVING THE STROLLER	31
13.	GETTING IN AND OUT OF A STROLLER	32
14.	CORRECT POSITION IN STROLLER.....	32
15.	USE OF THE STROLLER.....	32
16.	MOVING ON INCLINED SURFACES	33
17.	OVERCOMING THRESHOLDS OR CURBS.....	33
18.	TRANSPORT IN THE CAR AND IN OTHER VEHICLES.....	33
19.	CLEANING AND CARE	33
20.	STORAGE OF THE STROLLER.....	34
21.	SERVICE AND PERIODIC INSPECTIONS	34
22.	UTILIZATION	35
23.	COMPLIANCE WITH THE REQUIREMENTS FOR MEDICAL DEVICES.....	35
24.	LIST OF AUTHORIZED SERVICE POINTS	36
25.	WARRANTY CARD.....	37
26.	NOTES.....	38

1. PURPOSE AND INDICATIONS FOR USE

The special needs stroller ALFA is a class I medical device indicated for use by disabled people who, due to their health condition, cannot walk, stand or sit down independently. The product is intended for the transport and movement of the above persons in a sitting or lying position and for posture correction and this is its intended use.

Clinical indications include cerebral palsy, meningeal hernia, dystrophies, paresis, etc.

The product is intended for users 100-150cm height and up to 55 kg of weight. The product should be used in accordance with the recommendations of the attending physician or physiotherapist.

Contraindications:


In case of contraindication to the sitting position and/or passive upright positioning, the adjustment of the stroller should be consulted with a physiotherapist or attending physician.

2. PLATES







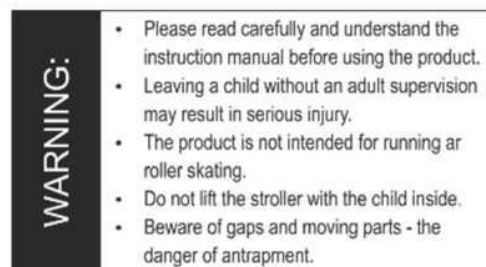

Nameplate



Warning plate



4. SYMBOLS DESIGNATION

	Product serial number		The manufacturer's name
	Permitted total user weight		The product is a medical device
	Production date		DESCRIPTION "WARNING!" and the symbol in this instructions are used to enhance the reader's attention to the content marked with this sign. Non-compliance with content marked with this description may endanger the health and safety of the user and accompanying persons, and may also damage the product.
	Conformity assessment mark according to applicable European regulations and directives		
	Warning about proceeding in accordance with the product Instructions for use		

5. IMPORTANT INSTRUCTIONS AND SAFETY RULES

- It is forbidden to use the stroller in a way for which the stroller is not intended.
- The product contains small parts that can be dangerous if swallowed by a child!
- **Due to the high adjustability of the stroller, during the angle adjustment of the seat and the angle adjustment of the backrest, it is possible to set such a position in which the user head will be below the pelovic line. It is forbidden to use such a position, as it could lead to the danger of the user swallowing or choking.**
- It is prohibited to place additional weight on the stroller, which in particular applies to placing additional weight on the back of the backrest, e.g. a backpack or bag. This may cause the stroller to tip over and injure the user! Does not apply to original additional accessories dedicated to the ALFA stroller that do not exceed the permissible load. All additional elements attached to the stroller affect its stability.
- **ATTENTION!** Adjustments are made without a load (user). While using, folding and adjusting of stroller mechanisms, there may be a risk of entrapment and/or squeezing some parts of the user body/caregiver in wholes/gaps between parts.
- **ATTENTION!** Depending on the adjustments made – applies to the setting of the push handle of the stroller, backrest height adjustment, backrest angle adjustment, seat depth adjustment, seat angle adjustment, footrest angle adjustment, and footrest adjustment – then this stroller may exceed to recommended maximum dimensions.
- **ATTENTION!** When passing through doors or similar restrictions, limiting the direction of travel of the stroller, make sure that there is enough space on both sides of the stroller to prevent hands or other body parts from getting pinched, as well as to prevent damage to the stroller.
- **ATTENTION!** The temperature of the stroller may increase when the stroller is exposed to sunlight.

WARNING!



- Never leave user unattended in a stroller, as it risks danger!
 - This product is not suitable for running, skating or roller-skating.
 - Beware of gaps and moving parts – risk of jamming.
 - Do not carry the stroller with the user inside it.
 - The stroller is packed in a plastic bag. Keep it away from infants and young children.
-
- **ATTENTION!** It is forbidden to use the stroller if the technical condition of the stroller or a single component raises doubts in the user's mind or is clearly abnormal, then it must not be used. In such a situation, it is recommended to urgently contact the Seller for advice.

6. GENERAL CONSTRUCTION OF THE STROLLER

Basic equipment



- 1- Canopy with window
- 2- Locking the frame folding mechanism
- 3- Cup holder
- 4- Safety barrier
- 5- Angle adjustment mechanism of the whole footrest
- 6- Footrest length adjustment
- 7- Rear wheel quick release mechanism
- 8- Front wheels directional lock
- 9- Headrest
- 10- Thoracolumbar pads
- 11- Five-point belts
- 12- Hip pads
- 13- Abduction and stabilization belts
- 14- Removable back and seat upholstery
- 15- Adjustable footrest with straps
- 16- Front swivel wheels

- 17- Backrest side elements
- 18- Backrest height adjustment
- 19- Transport lock
- 20- Push handle for the stroller
- 21- Angle adjustment mechanism for the push handle
- 22- Backrest angle adjustment
- 23- Frame folding mechanism
- 24- Backrest angle adjustment mechanism
- 25- Seat depth adjustment
- 26- Seat angle adjustment
- 27- Rear wheel amortization system
- 28- Tilt foot
- 29- Central brake
- 30- Rear wheels

Accessory equipment

- 1- Sunshade umbrella
- 2- Shopping basket
- 3- Sleeping bag for legs
- 4- Rain cover
- 5- Mosquito net

- 6- Wedge
- 7- Therapeutic table
- 8- Winter gloves
- 9- Backpack

7. STROLLER FUNCTIONS

7.1. Unfolding the stroller



1

Pull down the transport lock



2



3



4

The backrest assembly mechanism is located at the back of the seat.




5

Slide in the backrest



6

Unlocked mechanism set at the height of the holes on both sides

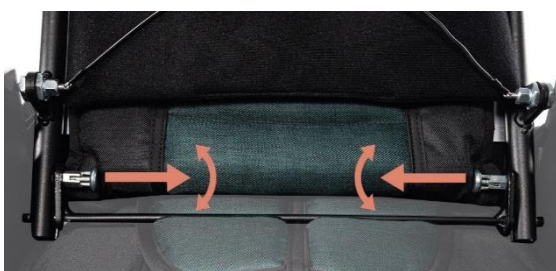

7

Slide the pins into the holes on both sides



8

Lock the backrest attachment mechanism



9

Pull the handle up and adjust the backrest angle



10

7.2. Folding the frame of the stroller



1




2

Unlock the backrest attachment mechanism



3

Lower the mechanism down

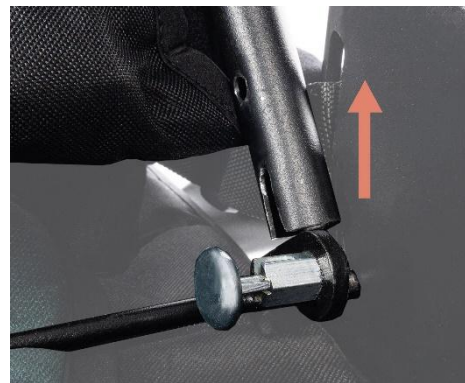

4

Remove the backrest



Press the red button



Simultaneously pull the handles on both sides of the frame upwards



Attach the transport lock

7.3. Transport lock. Putting on and taking off the transport lock



Lock applied




Removed lock

7.4. Front swivel wheels. Front wheels directional lock

Locking the front wheels




Unlocking the front wheels

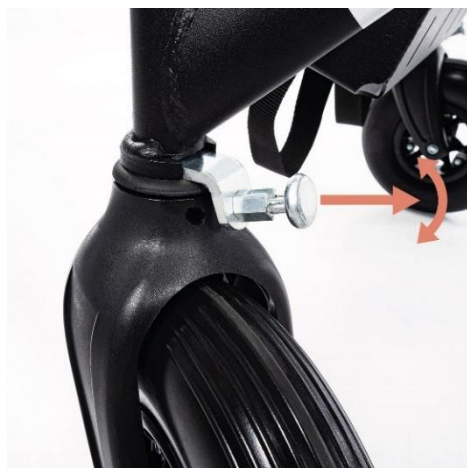

7.5. Rear wheels. Assembly and removal of rear wheels on the axle

Wheel assembly

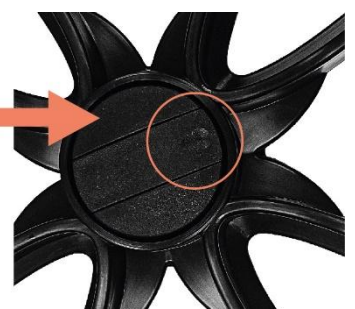



Removal of wheels



7.6. Rear wheel amortization system



Hard suspension



Soft suspension

7.7. Push handle for the stroller

Angle adjustment mechanism for the push handle



Press the buttons on both sides and change the angle of the push handle.

Angle adjustment of the push handle



7.8. Central brake



The central brake is used only for total wheel locking – there is no dynamic braking function.

NOTE! The brake condition should be checked regularly and any dirt and mud removed.

7.9. Tilt foot




Using a tilt foot makes it easier to drive the stroller up small thresholds or curbs.


7.10. Mechanism for attaching the backrest to the frame of the stroller

Assembly the backrest to the frame


- 1




The backrest assembly mechanism is located at the back of the seat.
- 2




Slide in the backrest
- 3




Unlocked mechanism set at the height of the holes on both sides
- 4

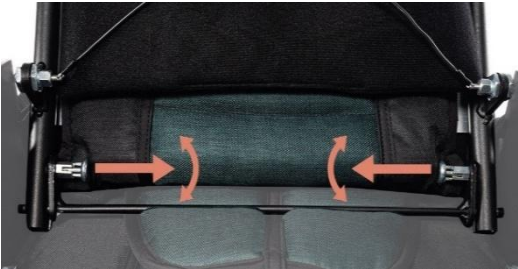

Slide the pins into the holes on both sides
- 5



Lock the backrest attachment mechanism
- 6



Pull the handle up and adjust the backrest angle


Removing the backrest from the frame



Unlock the backrest attachment mechanism



Lower the mechanism down

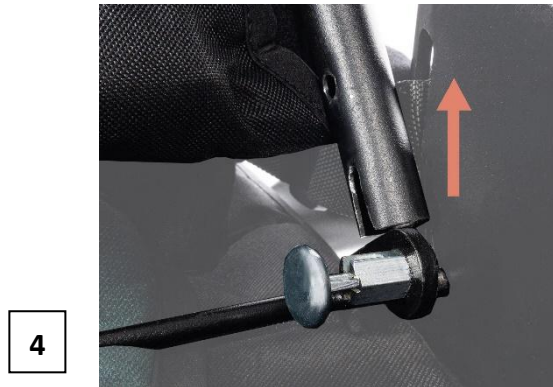

Remove the backrest

7.11. Seat depth adjustment



Minimum seat depth



Maximum seat depth


Pull the seat toward you or slide it in.



ATTENTION! To adjust the seat depth, use an Allen key (included).

7.12. Frame adjustment for changing seat angle

Seat angle adjustment mechanism

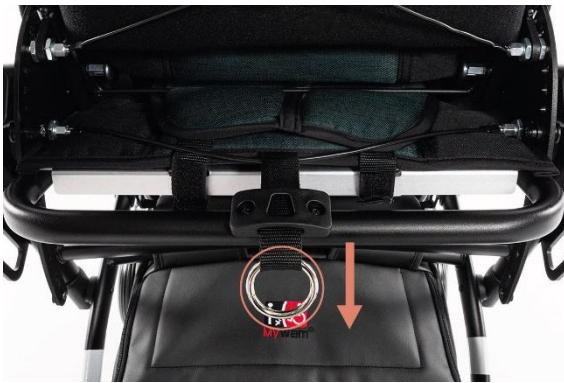

Make sure the pins have snapped on both sides of stroller.

Seat angle adjustment



7.13. Backrest height adjustment



Minimum setting heights
of the backrest



Maximum setting heights
of the backrest




Button to adjust heights
of the backrest



ATTENTION! To adjust height of the backrest, use an Allen key (included).

7.14. Frame adjustment of the backrest tilt angle

Backrest angle adjustment mechanism

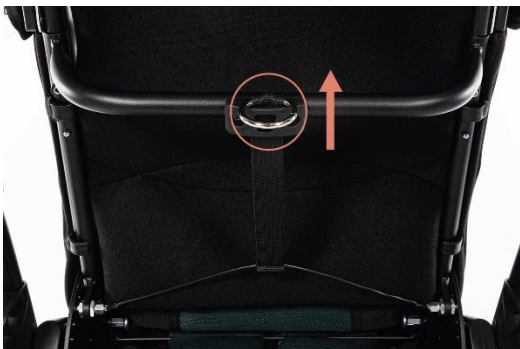

Attention! Every time you change the angle of the backrest, you need to readjust the length of the five-point belts.

Backrest angle adjustment



7.15. Footrest

Footrest height adjustment



Minimum height



Maximum height

Footrest angle adjustment



Press the buttons on both sides and change the angle of the footrest.



Attention! When transporting, storing, rehabilitation, getting in and out of the stroller, the footrest plate must be folded.


7.16. Five-point belts



Attention! When removing the backrest, detach the belts at the marked place.

Remember to fasten the belts when assembling the backrest.

Assembling the belts to the stroller

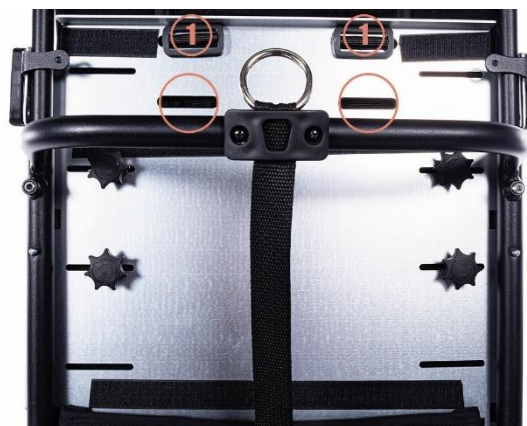

Seat belts assembling location on the back of the backrest

Pull the buckle through the holes in the backrest.



Pull the straps through the holes and thread through the buckle on the belt.



Bolt for assembling seat belts when the Wedge is not assembled.

The place where belts are assembling to the seat (where the wedge is assembled)

7.17. Headrest




7.18. Adjustable thoracolumbar pads support

Pads length adjustment




After installing the pads, secure the screws with the material.

Backrest width adjustment

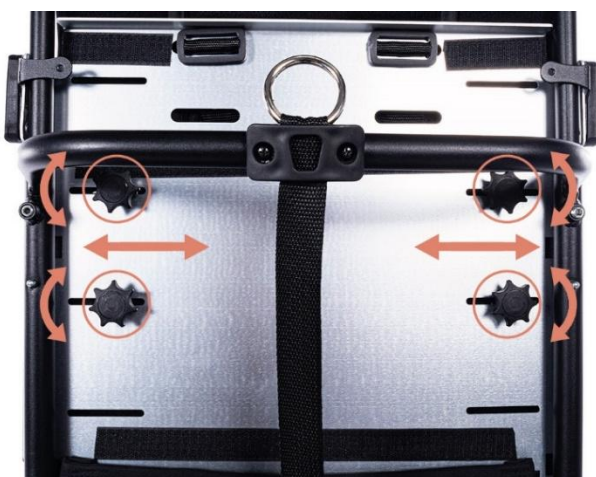

Pads height adjustment

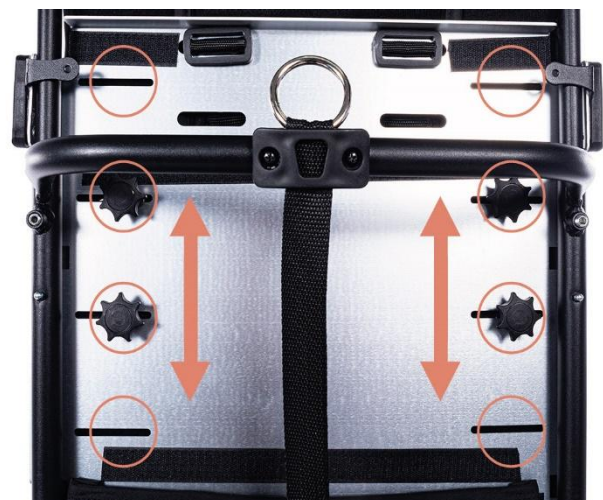


7.19. Adjustable hip pads


Pads height adjustment



Seat width adjustment

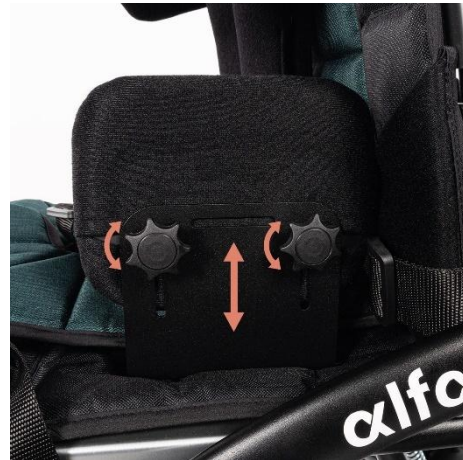

Seat depth adjustment

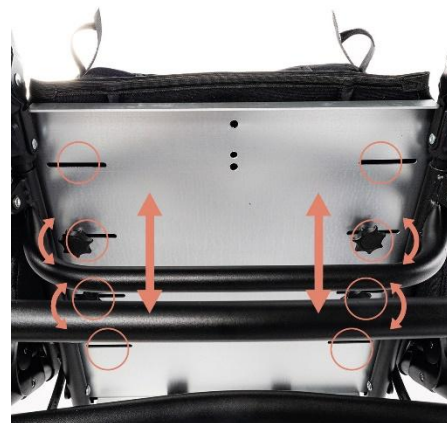

7.20. Backrest side elements



1



2



3

7.21. Abduction and stabilization belts



Fastening the belts to the seat



Fastening the belts to the frame



7.22. Safety barrier



Press the button on both sides and slide the safety barrier in or out.

7.23. Adjustable canopy with window

Mechanism for assembling the canopy to the frame of the stroller



Canopy attachment on stroller frame



Canopy attachment

Slide the mount on the canopy into the mount on the frame of the stroller.

Removal with the red button inside the mount on the canopy on both sides.

Changing the size of the canopy area



7.24. Cup holder



8. ADDITIONAL EQUIPMENT

8.1. Adjustable therapy table (put in place of safety barrier)



The maximum load is 3kg.

8.2. Backpack



The maximum load of the backpack is 2kg.

8.3. Shopping basket



The maximum load of the shopping basket is 3kg.

8.4. Winter gloves



8.5. Wedge



8.6. Rain cover (put on adjustable canopy with window)



8.7. Sleeping bag for legs (worn on the safety barrier)



8.8. Mosquito net (attached to adjustable canopy with window)



8.9. Sunshade umbrella



9. TECHNICAL SPECIFICATIONS OF THE ALFA STROLLER

length of the stroller	100-145 cm
width of the stroller	66 cm
height of the stroller	107-128 cm
total weight	26 kg
effective seat depth	30-40 cm
seat width	27-32 cm*, 41 cm**
backrest height	59-74 cm
backrest width	27-32 cm*, 40 cm**
distance between footrest and the seat / number of adjustments	30-40 cm/ 6 positions
length of the stroller folded for transport	103 cm
width of the stroller folded for transport	66 cm
height of the stroller folded for transport	58 cm
capacity of the shopping basket	3 kg
carrying capacity of a backpack	2 kg
maximum user weight	55 kg
maximum user height	100-150 cm
seat plane angle / number of adjustments	ok 5.7°-30.1° / 6 positions
backrest tilt angle / number of adjustments	ok -5.3°-91.1° / 8 positions
footrest angle / number of adjustments	ok 72°-193° / 8 positions

NOTES: *) values given with pads, ** values given without pads

Tolerance of measurement values +/- 20mm, +/- 1.5kg, +/- 1°

The manufacturer reserves the right to make changes to the technical parameters.

9.1. Material composition of the stroller

Material composition of the stroller: powder-coated steel and aluminum, fabrics 100% polyester and plastic (PP, PE).

10. DELIVERY AND TOOLS

The original packaging should include the following items:



- Instructions for use with warranty card,
- Additional equipment according to the order.

Before use:

- a. Make sure that the package has not been damaged. If it is found that the package is damaged, then at the presence of the courier, unpack the goods and check whether none of the stroller elements hasn't been damaged.
- b. On the basis of the specification check the package content (in case of any missing elements contact your Seller).

No tools are required to assemble and disassemble the ALFA stroller.

To adjust the backrest height and seat depth, use the 4mm Allen key included in the package.

11. ASSEMBLY AND DISASSEMBLY OF THE STROLLER

The product pre-packaged in the original carton is already pre-assembled.

- Complete the installation by performing the following steps:
 - unfold the stroller frame and install the backrest by adjusting the following parameters:
 - enable / disable the directional lock of the front wheels,
 - adjust the height and angle of the handle to push the stroller,
 - set the angle of the backrest,
 - adjust the depth of the seat,
 - adjust the length of the five-point belts,
 - adjust the footrest to the desired height,
 - install a safety barrier,
 - install a canopy with a window,
 - adjust the abduction and stabilization belts,
 - install and adjust: headrest, thoracolumbar pads, hip pads, backrest side elements,
 - install the appropriate additional equipment:
 - wedge,
 - backpack,
 - shopping basket,
 - sleeping bag for legs,
 - mosquito net,
 - rain cover,
 - sunshade umbrella,
 - cup holder.
- In order to disassemble the stroller, it is necessary:
 - remove the additional equipment,
 - remove the backrest from the stroller frame (see section 6.10),
 - remove the rear wheels from the stroller frame,
 - fold the frame of the stroller.

12. PREPARATION FOR USE

- Before using the stroller, ensure that the user footwear is comfortable and stabilizes the ankle joint well.

- Before using the stroller, ensure that the user is not placed in the stroller without clothing to cover the body in areas of contact with the upholstery of the stroller. This is to prevent abrasions to the user body and for hygienic reasons.
- For users with incontinence problems, put a special oilcloth on the seat of the stroller, which can be purchased at pharmacies or medical stores.
- In the case of using any additional aids, such as vents, drains, bags, catheters, etc., make sure that they are well protected against falling out, pulling out, tightening and not interfering with the use of the stroller.

In order to start using the stroller properly, you need to follow all the steps listed below:

- Place the stroller on a flat and safe surface.
- Properly unfold the stroller (see section 6.1).
- Apply the central brake.
- Make all adjustments to the stroller (see section 6. Stroller functions):
 - a) enable / disable the directional lock of the front wheels,
 - b) adjust the height of the push handle the stroller,
 - c) adjust the depth of the seat,
 - d) adjust the usable width of the seat,
 - e) adjust the seat tilt angle,
 - f) adjust the height of the backrest,
 - g) adjust the angle of the backrest,
 - h) adjust the length the five-point belts,
 - i) adjust the height of the footrest,
 - j) adjust the angle of the footrest tilt,
 - k) adjust the headrest,
 - l) adjust the thoracolumbar pads,
 - m) adjust the hip pads,
 - n) adjust the abduction and stabilization belts,
 - o) adjust the canopy with window,
 - Remove safety barrier and therapy table, if this equipment is available and mounted on the stroller.
 - Prepare to put on the five-point belts, footrest straps, and the abduction and stabilization belts.
 - Place the user in the stroller and make sure that all adjustments made beforehand are appropriate and safe for the user.
 - Put on the user five-point belts and footrest straps.
 - Fit the user with other necessary equipment in the stroller, if available.

13. MOVING THE STROLLER

The best way to move a stroller is to drive it using its wheels.

In cases where this is not possible (for example, to transport the stroller in a car), then for safety reasons, the stroller can be lifted and carried with both hands after removing the accessories to reduce the weight to be carried and folding it.

Use extreme caution when moving or lifting the frame of the stroller. The frame should be moved after folding and attaching the transport lock.

Do not use additional accessory to move the stroller – you risk damaging it.

14. GETTING IN AND OUT OF A STROLLER

When getting in and out of the stroller, place the stroller as close as possible to a stable and safe place to which the user will be changing their position. Once you are sure that the central brake is active and the footplate is lifted up to avoid standing on it, the user can change their position.

Do not stand on the footrest as there is a risk of the stroller tipping over, injuring the user and damaging the product.

Make sure that when getting in and out of the stroller, the user and other people should not stand on the footrest, as this may damage it and injure the user!

15. CORRECT POSITION IN STROLLER

For safety and health reasons, it is recommended that the user places the hips and back as close to the backrest as possible for even support of the spine (optimum support for all parts of the spine so that no free space is created).

The thighs should be supported as much as possible, along their entire length, which is helped by adjusting the depth of the stroller seat, and the feet should be rested on the footrest plate, which prevents sliding, maintains optimal tension in the leg muscles, and transfers some of the weight from the higher parts of the body.



ATTENTION! Due to the high adjustability of the stroller, during the angular adjustment of the seat and angular adjustment of the backrest, it is possible to set such a position in which the user head is below the pelvic line. **It is forbidden to use such a position**, as it could lead to the danger of choking or choking of the user.



Prohibited positioning of the stroller

16. USE OF THE STROLLER

When using the stroller, always hold the handle to push the stroller to prevent the stroller from rolling inertly. Release the central brake while holding the push handle of the stroller and push the stroller in a safe direction.

17. MOVING ON INCLINED SURFACES

Always position the stroller facing forward when climbing hills or ramps. Limit your speed on inclined surfaces and move as slowly as possible.

The person pushing the stroller must have appropriate strength to control the stability of the stroller.

Always keep in mind that the position of the center of gravity of the stroller changes when traversing uneven or sloping terrain. In such a case, be very careful and delay the stroller.

When using ramps, make sure that the ramp is suitable and the wheels of the stroller are high enough to negotiate the ramp, that the frame of the stroller will not come into contact with the ramp, and that a second person is available to assist, as there is a risk of the stroller tipping over, damaging the product and injuring the user.

18. OVERCOMING THRESHOLDS OR CURBS

When negotiating small obstacles (small thresholds, small curbs, etc.), press the tilt foot and push handle of the stroller downward and toward you with your foot to lift the front wheels of the stroller and carefully drive the front wheels onto the obstacle. Then slowly pull the push handle of the stroller upward from the ground to lift the rear wheels of the stroller and carefully drive the rear wheels on the obstacle.

19. TRANSPORT IN THE CAR AND IN OTHER VEHICLES

- Remove all accessories from the stroller and store them in a safe place. Fold the stroller and place it in the trunk.

If the dismantled stroller is not in the trunk of the car separate from the passenger compartment, the stroller frame must be carefully secured to the vehicle. For this purpose, you can use unused seat belts.

- In order to transport a stroller in other vehicles, such as trains, trams, buses, planes, ships, etc., it is necessary to ask the people in charge of transportation in these means of transport about the conditions for safe travel to the destination with a stroller and relevant requirements.



The ALFA stroller should not be used as a seat to transport an user in a vehicle.

20. CLEANING AND CARE

Upholstery is machine washable at up to 40°C. Prohibition of spinning. Upholstery should be dried on the stroller structure.

The shopping basket should be washed by hand (the stain of the stroller should be wiped with a damp cloth). Stains should be removed as soon as they form on the surface of the shopping basket.

ATTENTION! Do not clean with chemicals containing solvents and bleach. The manufacturer is not responsible for damages caused by the use of inappropriate cleaning products.

Make sure the product is always dry before use.

When using detergents dedicated to this type of material - follow the instructions of the detergent use.



Wash at 40°



Do not bleach



Do not tumble dryer



Do not iron

21.STORAGE OF THE STROLLER

Clean stroller should be stored in a clean, dry and well-ventilated room, preferably at 0-35°C. For long-term storage, additionally follow the steps described in point 21 “every one month”.

22. SERVICE AND PERIODIC INSPECTIONS

For safe and trouble-free use of the stroller, it is necessary to regularly and thoroughly submit it to the inspections and servicing described below.

In a situation where the stroller user is not able to carry out the above-mentioned activities on their own, they should return the stroller to a specialized stroller service point or directly to the Manufacturer, who will properly carry out the inspection and necessary servicing activities. These activities may be subject to an additional fee.

NOTE: The Manufacturer is responsible for defects or technical damage to the stroller caused only when the stroller is operated correctly. Responsibility for the proper operation of the stroller rests with the Manufacturer only if all recommendations of the Manufacturer are strictly followed.

NOTE: The Manufacturer is responsible for the proper functioning of the stroller only if original parts and original accessories are used in it.

All repairs should be carried out in specialized authorized Services (see point 24) or directly at the Manufacturer. Repairs to the stroller performed by anyone other than an authorized Service or the Manufacturer will void the warranty. The Manufacturer is not responsible for injury or damage of the product that may be a consequence of repairs performed by persons other than the authorized Service or the Manufacturer.

Service should be performed:

- **Each time before using the stroller**
 - Check the efficiency of the central brake - After activating the central brake, both rear wheels of the stroller should be locked and cannot rotate.
 - Check the attachment of structural components - Inspect all mechanisms of the stroller and its moving parts, and in particular, pay attention to: correct mounting of the backrest to the frame of the stroller, proper mounting of the wheels, proper functioning of the five-point belts, correct functioning of the seat angle adjustment mechanism, correct functioning of the backrest recline mechanism and proper mounting of the footrest.
 - Check the condition of the tires - Check that the tires on all four wheels are not damaged or worn.

- Inspection of solid frame parts, seat with backrest and footrest frame - Visually inspect the condition of the structural profiles. These components must not be damaged or cracked.
- **Every one month**
 - Cleaning of the stroller
 - Visual check of all the moving elements, bearings, adjusting mechanisms – lubricate with silicone grease.
 - Check all bolted connections for looseness.

23. UTILIZATION

Disposal and recycling: in accordance with country-specific regulations.

24. COMPLIANCE WITH THE REQUIREMENTS FOR MEDICAL DEVICES

Class I medical device in accordance with the Regulation of the European Parliament and of the Council (EU) 2017/745 of April 5, 2017 on medical devices. The manufacturer issued an appropriate declaration of conformity for the product, which is available at www.mywam.eu

25. LIST OF AUTHORIZED SERVICE POINTS

POLAND

MyWam Sp. z o.o.

Wiosny Ludów 91, 40-373 Katowice, Poland

tel: +48 32 733 11 31

website: www.mywam.eu

e-mail: mywam@mywam.eu

26. WARRANTY CARD

Series and number:

Purchase date: - -

Seller stamp and signature:

.....
.....
.....

Warranty conditions

- The Manufacturer (Guarantor) provides the Buyer with a warranty for the purchased stroller for a period of 24 months, counting from the date of purchase.
- **The buyer complaining about the stroller should deliver the stroller to the Seller in a package, clean (free of stains, additional equipment not belonging to the stroller, moisture, hair, etc.) together with the elements of the stroller protected against damage.**
- **Damage caused by insufficient protection of the stroller shipment and its components is not subject to warranty repairs.**
- The complainant (Buyer) may not demand that the Manufacturer repair the damage in the event of damage to the shipment if he incorrectly labels or incorrectly insures the shipment of the stroller and its components.
- The warranty period in the case of accepting the complaint is extended by the time of repair of the stroller, which is counted from the moment of handing over the stroller to the Seller under warranty until the day on which the stroller was ready to be collected.
- The activities provided for in the instructions for use, which the stroller user is obliged to perform on his own and at his own expense, cannot be performed as part of the warranty repair.
- Failure to use the stroller in accordance with the intended use, inconsistent with the instructions for use or inconsistent with the principles of proper operation will void the warranty.
- Any structural changes to the truck void the warranty (in the area of change or in its entirety).
- Damage to the stroller caused by: improper operation, improper maintenance, improper storage of the stroller, mechanical damage and natural abrasion and wear of parts (e.g. upholstery, wheels) are not covered by the warranty.
- The manufacturer warns against warranty repairs in unauthorized service facilities. Finding such a repair or unauthorized modification of the stroller will void the warranty.
- **The warranty card is valid provided that it has: the date of sale confirmed by the Seller's company stamp, the Seller's signature, the type and serial number of the truck.**
- The warranty card is valid provided that it does not contain deletions or corrections.
- The manufacturer of the stroller conducts a paid post-warranty service at its headquarters.
- The manufacturer specifies the expected service life of the product at 4 years. Further use after this period is possible, but taking into account the care for safety and comfort of use, we recommend that the manufacturer or authorized service center carry out a technical inspection. The inspection service is payable and includes an assessment of the technical condition of the device, safety verification and any necessary maintenance activities.
- In matters not covered by this warranty card, the provisions of the Civil Code (Poland) shall apply.
- All matters related to claims under the warranty or warranty of the stroller should be directed to the Seller.

27. NOTES