

MANUFACTURER / MARKETING AND ADVERTISING: MyWam Sp. z o.o.

CE  
mewa



ILLUSTRATIVE PHOTO



## INSTRUCTIONS FOR USE

Edition 15 / 2026-06-16

**Special needs stroller MEWA**

**THIS IS A MEDICAL DEVICE.**

**USE IT ACCORDING TO THE INSTRUCTIONS FOR USE OR LABEL.**

## PURPOSE OF THE INSTRUCTION

Instructions for use is designed for people who directly care of people with disabilities (assistants, parents, guardians), which will be transported in a stroller. It is also intended for doctors and physiotherapists operating the stroller.

**ATTENTION!** Before using the stroller, each User is obliged to read the instruction and act in accordance with its guidelines.

**ATTENTION!** In the event of any medical incident involving a MEWA stroller, the manufacturer and the competent authority of the member state in which the user or patient resides must be notified immediately in writing by registered letter or e-mail with acknowledgment of receipt.

Manufacturer contact details:

- postal address: MyWam Sp. z o.o., Plebiscytowa 3, 41-100 Siemianowice Śląskie, Poland
- contact phone number: +48 32 733 11 31
- e-mail address: [mywam@mywam.eu](mailto:mywam@mywam.eu)



Instructions for use version: EN **Edition 15 / 2026-06-16**

The instruction was translated using Microsoft Translator.

MyWam Sp. z o.o. reserves the right to make technical and commercial changes to the contents of the instructions for use without prior notice. Each change will be determined by the date the instructions were last updated. The electronic version of the instruction is available at [www.mywam.eu](http://www.mywam.eu)

**ATTENTION!** Pictures given in the instructions for use are for reference only and can vary from the purchased model.

MyWam Sp. z o.o. has implemented a quality management system compliant with the ISO 13485 standard in the design, production and sale of class I medical devices. System certified by LL-C (Certification).



**TABLE OF CONTENTS**

|                                                                         |    |
|-------------------------------------------------------------------------|----|
| PURPOSE OF THE INSTRUCTION.....                                         | 2  |
| TABLE OF CONTENTS .....                                                 | 3  |
| 1. PURPOSE AND INDICATIONS FOR USE .....                                | 5  |
| 2. PLATES .....                                                         | 5  |
| 3. DESIGNATION OF SYMBOLS .....                                         | 5  |
| 4. IMPORTANT INSTRUCTIONS AND SAFETY RULES.....                         | 6  |
| 5. GENERAL CONSTRUCTION OF THE STROLLER .....                           | 7  |
| 6. STROLLER FUNCTIONS.....                                              | 9  |
| 6.1. Unfolding the stroller frame.....                                  | 9  |
| 6.2. Folding the frame of the stroller .....                            | 10 |
| 6.3. Seat .....                                                         | 10 |
| 6.4. Transport lock. Insertion and removal of the lock .....            | 13 |
| 6.5. Front swivel wheels. Front wheels directional lock.....            | 13 |
| 6.6. Rear wheels. Assembly and removal of rear wheels on the axle ..... | 14 |
| 6.7. Rear wheel suspension system .....                                 | 15 |
| 6.8. Push stroller handle.....                                          | 15 |
| 6.9. Central brake .....                                                | 16 |
| 6.10. Tilt foot .....                                                   | 16 |
| 6.11. Footrest .....                                                    | 16 |
| 6.12. Adjustable backrest.....                                          | 17 |
| 6.13. Five-points belt.....                                             | 18 |
| 6.14. Safety barrier.....                                               | 19 |
| 6.15. Adjustable canopy with window .....                               | 20 |
| 6.16. Abduction and stabilization belts.....                            | 21 |
| 6.17. Head adjustable pads.....                                         | 22 |
| 6.18. Torso pads .....                                                  | 22 |
| 6.19. Cushion pillows.....                                              | 23 |
| 6.20. Cup holder .....                                                  | 23 |
| 6.21. Seat minimizing kit .....                                         | 24 |
| 7. ADDITIONAL EQUIPMENT.....                                            | 24 |
| 7.1. Wedge.....                                                         | 24 |
| 7.2. Therapeutic table.....                                             | 25 |
| 7.3. Shopping basket .....                                              | 26 |
| 7.4. Travel bag .....                                                   | 26 |
| 7.5. Sun umbrella .....                                                 | 26 |
| 7.6. Winter gloves .....                                                | 27 |

|                                                                       |    |
|-----------------------------------------------------------------------|----|
| 7.7. Sleeping bag for legs (to be placed on the safety barrier) ..... | 27 |
| 7.8. Mosquito net (attached to adjustable canopy with window) .....   | 27 |
| 7.9. Rain cover (attached to adjustable canopy with window).....      | 28 |
| 8. TECHNICAL SPECIFICATION OF THE MEWA STROLLER.....                  | 28 |
| 8.1. Material composition of the stroller .....                       | 29 |
| 9. DELIVERY AND TOOLS.....                                            | 29 |
| 10. ASSEMBLY AND DISASSEMBLY OF THE STROLLER.....                     | 29 |
| 11. PREPARATION FOR USE .....                                         | 30 |
| 12. MOVING THE STROLLER .....                                         | 30 |
| 13. GETTING IN AND OUT OF A STROLLER .....                            | 31 |
| 14. CORRECT POSITION IN A STROLLER.....                               | 31 |
| 15. USE OF THE STROLLER .....                                         | 31 |
| 16. MOVING ON INCLINED SURFACES.....                                  | 31 |
| 17. OVERCOMING THRESHOLDS OR CURBS.....                               | 32 |
| 18. TRANSPORT IN THE CAR AND IN OTHER VEHICLES .....                  | 32 |
| 19. CLEANING AND CARE .....                                           | 32 |
| 20. STORAGE OF THE STROLLER.....                                      | 33 |
| 21. SERVICE AND PERIODIC INSPECTIONS.....                             | 33 |
| 22. UTILIZATION .....                                                 | 34 |
| 23. COMPLIANCE WITH THE REQUIREMENTS FOR MEDICAL DEVICES .....        | 34 |
| 24. LIST OF AUTHORISED SERVICE POINTS.....                            | 35 |
| 25. WARRANTY CARD .....                                               | 36 |
| 26. NOTES.....                                                        | 37 |

## 1. PURPOSE AND INDICATIONS FOR USE

The special needs stroller MEWA is a Class I medical device indicated for use by those disabled people who, due to their health condition, cannot walk, standing or sit up unaided. The product is intended for the transport and movement of the above persons in a sitting or lying position and for posture correction and this is its intended use.

Clinical indications include cerebral palsy, meningeal hernia, dystrophies, paresis, etc.

The device is intended for users aged between 3 and 10 years, with a height of 90-125 cm and a weight of up to 40 kg. The product should be used as recommended by the attending physician or physiotherapist.

### Contraindications:


If there is a contraindication to sitting and/or passive verticalisation, the adjustment of the stroller should be consulted with a physiotherapist or general practitioner.

## 2. PLATES

Nameplate



Warning plate

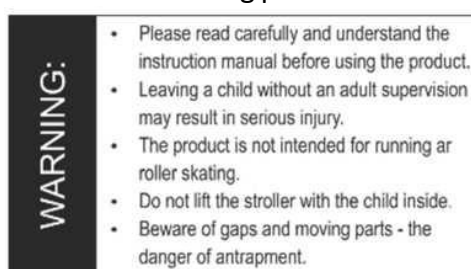

## 3. DESIGNATION OF SYMBOLS

|                                                                                     |                                                                                        |                                                                                     |                                                                                                                                                                                                                                                                                                                                             |
|-------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Product serial number                                                                  |  | Name of manufacturer                                                                                                                                                                                                                                                                                                                        |
|  | Permitted total weight user                                                            |  | The product is a medical device                                                                                                                                                                                                                                                                                                             |
|  | Production date                                                                        |  | The " <b>WARNING!</b> " descriptor and symbol in this instruction are used to reinforce the reader's attention to the content marked with this sign. Failure to comply with the content marked with this description may endanger the health and safety of the user and accompanying persons, as well as can lead to damage to the product. |
|  | Conformity assessment mark according to applicable European regulations and directives |                                                                                     |                                                                                                                                                                                                                                                                                                                                             |
|  | Warning of proceedings in accordance with the instructions for use of the product      |                                                                                     |                                                                                                                                                                                                                                                                                                                                             |

#### 4. IMPORTANT INSTRUCTIONS AND SAFETY RULES

- It is prohibited to use the stroller in a manner for which it is not intended.
- The product contains small parts which can be dangerous if swallowed by a child!
- It is forbidden to place additional weight on the stroller, in particular to place additional weight on the back of the backrest, e.g. a rucksack or bag. This may cause the MEWA stroller to tip over and injure the user! This does not apply to original accessories dedicated to the MEWA stroller that do not exceed the permitted load. All additional components attached to the stroller affect the stability of the stroller.
- **ATTENTION:** Never adjust the pram when the user is not secured with the five-point belts.
- **ATTENTION:** When using the stroller and when folding and adjusting the mechanisms, particular care should be taken with moving parts, as there may be a danger of the user/carer's body parts becoming trapped and/or pinched in the openings/gaps between components.
- **ATTENTION:** Depending on the adjustments made - concerning the setting of the push handle, backrest height adjustment, backrest angle adjustment, seat depth adjustment, seat angle adjustment, footrest angle adjustment and footrest setting - this stroller may exceed the recommended maximum dimensions.
- **NOTE:** When passing through doors or similar constrictions that restrict the direction of travel of the stroller, ensure that there is sufficient space on both sides of the stroller to prevent hands or other parts of the body from being trapped, as well as to prevent damage to the stroller.
- **NOTE:** The temperature of the stroller may increase when the stroller is exposed to sunlight.

#### WARNING!



- Never leave user unattended in a stroller, as it risks danger!
- This product is not suitable for running or skating or rollerblading.
- The stroller is not intended as a seat in a motor vehicle.
- Beware of crevices and moving parts - risk of jamming.
- Do not carry the stroller with the user.
- The stroller is packed in a plastic bag. Keep out of reach of babies and small children.

- **ATTENTION:** It is forbidden to use the stroller if the technical condition of the product or its single component raises doubts in the user's mind or is clearly incorrect, in which case it must not be used. In such a situation, it is advisable to contact the Seller urgently for advice.

## 5. GENERAL CONSTRUCTION OF THE STROLLER

### Basic equipment



- 1- Canopy with window
- 2- Head adjustable pads
- 3- Cushion pillows
- 4- Safety barrier
- 5- Torso adjustable pads
- 6- Abduction and stabilization belts
- 7- Adjustable footrest with straps
- 8- Seat minimizing kit
- 9- Five-points belt
- 10- Seat adjustment mechanism
- 11- Transport lock

- 12- Push handle
- 13- Cup holder
- 14- Backrest adjustment mechanism
- 15- Rear wheel suspension system
- 16- Tilt-foot
- 17- Central brake
- 18- Back and seat upholstery
- 19- Front wheel direction lock
- 20- Front wheels with swivel castors
- 21- Rear wheels

## Additional equipment



- 1- Therapeutic table
- 2- Sun umbrella
- 3- Mosquito net
- 4- Rain cover

- 5- Sleeping bag
- 6- Wedge
- 7- Winter gloves
- 8- Shopping basket
- 9- Travel bag

## 6. STROLLER FUNCTIONS

### 6.1. Unfolding the stroller frame

1



2



Put all the wheels on

3



Pull down the transport lock

4



Pull the handle towards you

5




## 6.2. Folding the frame of the stroller

1



Press the red button

2

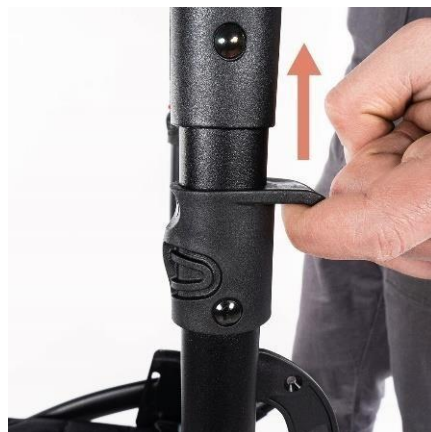

Simultaneously pull the handles on both sides of the frame upwards

3



4



## 6.3. Seat

### Ways of assembling the seat



## Mechanism for mounting the seat to the frame



## Seat assembly



Slide the metal bracket into the seat mounting mechanism



## Removing the seat



Press down on either side of the stroller on the red buttons and slide the seat out of the frame.

## Seat angle adjustment mechanism



Pull the handle towards you and change the seat angle.

## Angle adjustment of the seat



**NOTE:** Any change in the inclination of the seat changes the overall centre of gravity. This affects the stability of the stroller.

**NOTE:** Make sure the user's five-point belt is properly fastened to ensure body stabilization when changing the seat angle.

#### 6.4. Transport lock. Insertion and removal of the lock



1

Lock applied



2

Removed lock

#### 6.5. Front swivel wheels. Front wheels directional lock

##### Wheel assembly



1

Dismantling the wheels



2



1



2

### Locking the front wheels



### Unlocking the front wheels



## 6.6. Rear wheels. Assembly and removal of rear wheels on the axle

### Wheel assembly



### Dismantling the wheels



## 6.7. Rear wheel suspension system



Hard suspension



Soft suspension

## 6.8. Push stroller handle

Angle adjustment mechanism for the push handle



Press the buttons on either side and change the angle of the push handle.

Angle adjustment of the push handle



## 6.9. Central brake



The central brake is used only and exclusively to completely lock the wheels - it does not have a dynamic braking function.

**NOTE!** The brake condition should be checked regularly and any dirt and mud removed.

## 6.10. Tilt foot



Using the tilt foot it easier to drive the stroller up small thresholds or curbs.

## 6.11. Footrest

### Footrest height adjustment



Minimum height



Maximum height

## Footrest angle adjustment




Press the buttons on either side and change the angle of the footrest.

## 6.12. Adjustable backrest



**Note:** During transport, storage, rehabilitation, getting in and out of the stroller, the footrest must be folded down.

## Backrest angle adjustment mechanism

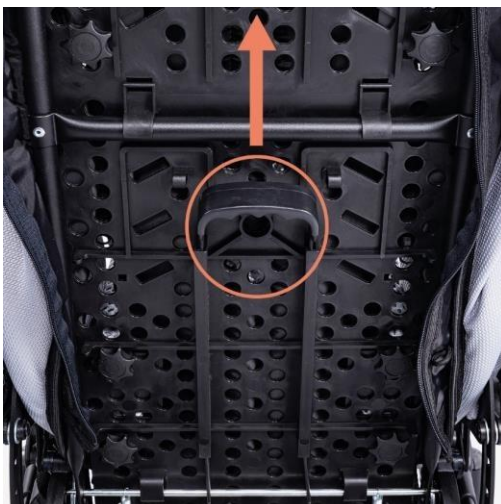



2

Backrest angle adjustment




**Note:** Every time the angle of the backrest is changed, the length of the five-points belt must be readjusted.

**NOTE:** Make sure the user's five-points belt is properly fastened to ensure body stabilization when changing the backrest angle.

### 6.13. Five-points belt



## Adjusting the belts

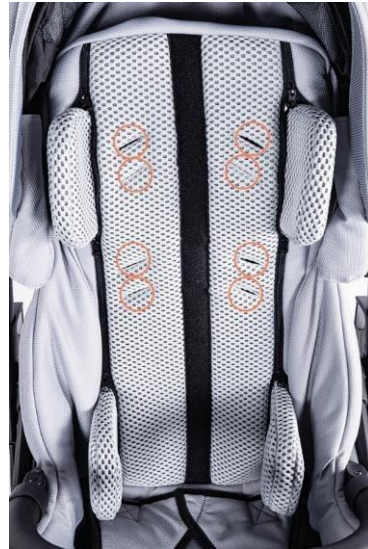

## 6.14. Safety barrier



### Barrier assembly



1





2

## Barrier dismantling



## 6.15. Adjustable canopy with window

### Mechanism for assembling the canopy

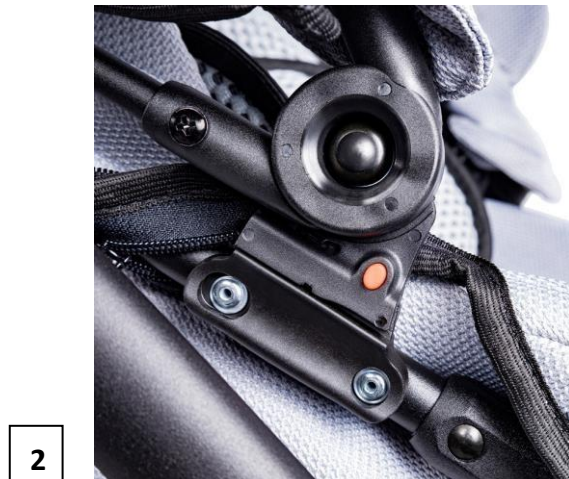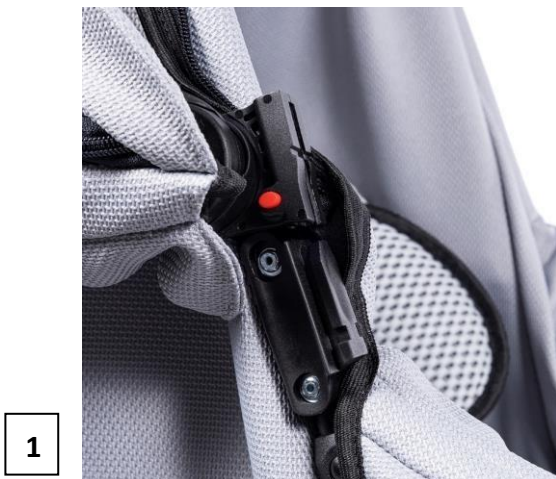

Slide the attachment on the canopy into the attachment on the frame of the stroller.

Removable by means of a red button on the outside of the canopy attachment on both sides.

## Changing the size of the canopy area



## Window with mosquito net



## 6.16. Abduction and stabilization belts




### 6.17. Head adjustable pads




#### Pads adjustment



### 6.18. Torso pads

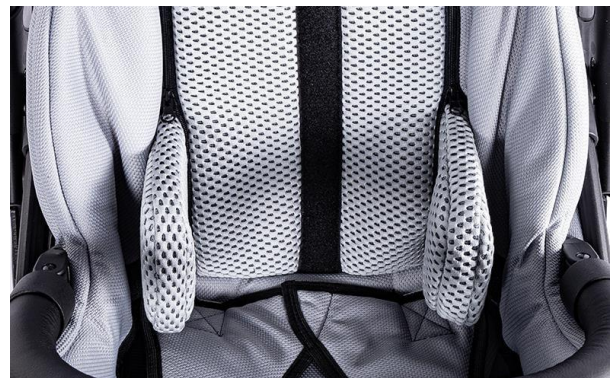

## Pads adjustment

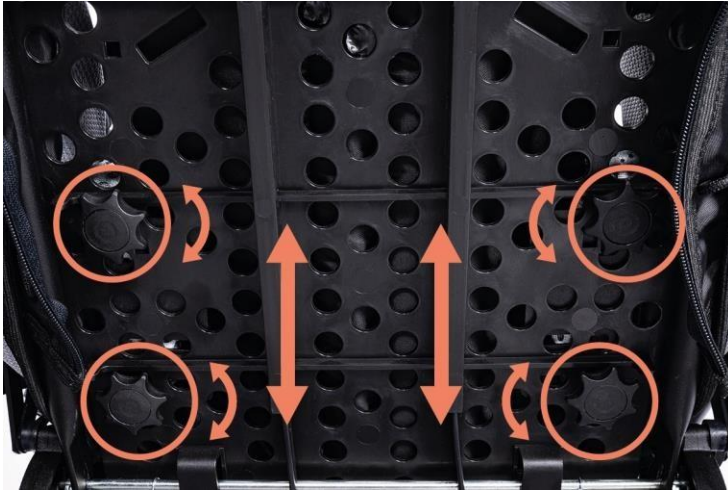

### 6.19. Cushion pillows



Clip on the backrest of the stroller when not wearing other accessories.

### 6.20. Cup holder



### 6.21. Seat minimizing kit



## 7. ADDITIONAL EQUIPMENT

### 7.1. Wedge

Wedge has a two-stage adjustment level.



## 7.2. Therapeutic table



Maximum load is 3kg.

### Table assembly



### Table disassembly



### 7.3. Shopping basket



Maximum load is 3kg.

### 7.4. Travel bag



Maximum load bag is 2kg.

### 7.5. Sun umbrella



### 7.6. Winter gloves



### 7.7. Sleeping bag for legs (to be placed on the safety barrier)



### 7.8. Mosquito net (attached to adjustable canopy with window)



## 7.9. Rain cover (attached to adjustable canopy with window)



## 8. TECHNICAL SPECIFICATION OF THE MEWA STROLLER

|                                                                |                                          |
|----------------------------------------------------------------|------------------------------------------|
| length of the stroller                                         | 97-132 cm                                |
| width of the stroller                                          | 62.5 cm                                  |
| height of the stroller                                         | 125 - 128 cm                             |
| total weight                                                   | 20.5 - 25 kg                             |
| effective seat depth                                           | 24*/28 cm                                |
| seat width                                                     | 25*/32 cm                                |
| backrest height                                                | 62 cm                                    |
| backrest width                                                 | 20** / 30 cm                             |
| distance between footrest and the seat / number of adjustments | 17-32 / 5 positions                      |
| length of the stroller folded for transport                    | frame 84 cm /seat 88 cm                  |
| width of the stroller folded for transport                     | frame 61 cm /seat 43 cm                  |
| height of the stroller folded for transport                    | frame 26 cm /seat 42 cm                  |
| capacity of the shopping basket                                | 3 kg                                     |
| carrying capacity of a backpack                                | 2 kg                                     |
| maximum user's weight                                          | 40 kg                                    |
| maximum user's height                                          | 90-125 cm                                |
| seat plane angle / number of adjustments                       | approx. 6.6°-22.6° / 4 positions         |
| backrest tilt angle / number of adjustments                    | approx. 9°-87° / 4 positions + reclining |
| footrest angle / number of adjustments                         | approx. 64°-180° / 5 positions           |

NOTES: \*) values given with pads, \*\* values given without pads

Tolerance of measurement values +/- 20mm, +/- 1.5kg, +/- 1°

The manufacturer reserves the right to make changes to the technical parameters.

### 8.1. Material composition of the stroller

Material composition of the stroller: powder-coated steel and aluminum, fabrics 100% polyester and plastic (PP, PCV, PA, PU).

## 9. DELIVERY AND TOOLS

The original packaging should include the following items:

- Folded stroller frame with seat and backrest, footrest, wheels, canopy, pads, cup holder, safety barrier, cushions pillow, seat minimizing kit,
- Additional equipment according to the order,
- Instructions for use with warranty card.

Before use:

- a. Make sure that the package has not been damaged. If it is found that the package is damaged, then at the presence of the courier, unpack the goods and check whether none of the stroller elements hasn't been damaged.
- b. On the basis of the specification check the package content (in case of any missing elements contact your Seller).

No tools are required to assemble or disassemble the MEWA stroller, except for adjusting the height of the footrest, where a Phillips screwdriver and an 8mm flat wrench are required.

## 10. ASSEMBLY AND DISASSEMBLY OF THE STROLLER

• **Assembly of the stroller.** The product is factory-packed in the original carton and is already pre-assembled. Complete the assembly by performing the following steps:

- a) mount the front wheels,
- b) mount the rear wheels onto the axle,
- c) unfold the stroller frame with the seat by adjusting the parameters:
  - o engage/disengage the front wheel direction lock,
  - o adjust the height and angle of the handle for pushing the stroller,
  - o set the seat inclination angle,
  - o set the backrest inclination angle,
  - o adjust the length of the five-point belts,
  - o adjust the height and angle of the footrest,
  - o adjust the abduction and stabilization belts,
  - o install the safety barrier,
  - o adjust the canopy with a window,
  - o adjust the headrest pads and torso pads,
  - adjust the width and usable depth of the seat (seat minimizing kit),
- d) fit the appropriate equipment and adjust the parameters, for example:
  - o install a therapeutic table,
  - o install and adjust the wedge,
  - o install a travel bag,
  - o install a shopping basket,

o install selected equipment dedicated to the seasons: a sleeping bag, winter gloves, a mosquito net, a rain cover, an umbrella.

- **Dismantling the stroller:**

- a) remove the additional equipment,
- b) unfasten the seat,
- c) block the front wheels, fold the footrest and the stroller frame,
- d) unfasten the rear wheels,
- e) fold the stroller by closing it with the transport lock.

## 11. PREPARATION FOR USE

Before using the stroller, you should:

- o Check that the stroller frame has been correctly unfolded and that the frame folding mechanism has been correctly closed (locked).
- o Check that the seat has been correctly mounted to the stroller frame and that the seat attachment mechanism to the stroller frame has been correctly closed (locked).
- o Correctly make all adjustments and adjust the stroller to the user's needs, including setting the appropriate depth of the therapeutic seat, the appropriate inclination of the therapeutic seat and the inclination of the backrest, set the appropriate inclination of the footrest, adjust the appropriate length of the five-point belts. Check that all adjustment elements are properly seated and secured.
- o Check that the stroller upholstery is dry.
- o Secure the user with all available stabilizing elements, such as: five-point belts, narrowing and shallowing inserts, abduction and stabilizing belts, wedge, footrest, footrest straps, safety barrier or therapeutic table. This is to protect the user from falling out and sliding out of the stroller and to protect the user's feet from falling off the footrest. Make sure that all adjustments made earlier are appropriate and safe for the user.
- o Make sure that the user is not placed in the stroller without clothing that covers the body in places of contact with the stroller upholstery. This is to prevent abrasions to the user's body and for the hygiene.
- o In the case of user's who have problems with holding urine, a special oilcloth should be placed on the stroller seat, which can be purchased in pharmacies or medical stores.
- o In the case of using any additional aids such as venflons, drains, bags, catheters, etc., make sure that they are well secured against falling out, tearing out, squeezing and do not interfere during use of the stroller.

## 12. MOVING THE STROLLER

The best way to carry a stroller is to push it using its wheels.

In cases where this is not possible (e.g. to transport the stroller in a car), for safety reasons, the stroller can be lifted and carried with both hands after previously removing the additional equipment to reduce the carrying weight and folding it.

When carrying or lifting the stroller frame, special care should be taken. The frame should be carried after it has been folded and the transport lock has been fitted.

Do not use additional equipment to carry the stroller - it may be damaged.

### 13. GETTING IN AND OUT OF A STROLLER

When getting in and out of the stroller, place the stroller as close as possible to a stable and safe place to which the user will be changing their position. After making sure that the central brake is engaged and the footrest is raised to avoid standing on it, the user may transfer.

**WARNING!** Do not stand on the footrest, as there is a risk of the stroller tipping over, injury to the user and risk of damage to the product.

Please ensure that when getting in or out of the stroller, the user and any other person do not stand on the footrest, as this may damage it and injure the user!

### 14. CORRECT POSITION IN A STROLLER

For safety and health reasons, it is recommended that the user has the hips and back positioned as close to the backrest as possible in order to support the spine evenly (optimal support of all parts of the spine so that no free space is created).

The thighs should be supported as much as possible, along their entire length, which is achieved by adjusting the depth of the stroller seat and the feet should rest on the footrest plate, which prevents slipping, maintains optimum muscle tone in the legs and transfers some of the weight from the upper body.

### 15. USE OF THE STROLLER

During use, always hold the push handle to prevent the stroller from rolling away.

Release the central brake while holding the push handle and push the stroller in a safe direction.

### 16. MOVING ON INCLINED SURFACES

Always position the stroller facing forward when climbing hills or ramps. Limit your speed on inclined surfaces and move as slowly as possible.

The person pushing the stroller must have appropriate strength to control the stability of the stroller.

Always keep in mind that the position of the centre of gravity of the stroller changes when traversing uneven or sloping terrain. In this case, special care must be taken and the stroller must be belayed.

When using ramps, ensure that the ramp is suitable, that the wheels of stroller are high enough to negotiate the ramp, that the stroller frame does not come into contact with the ramp and that a second person is provided to assist as there is a risk of the stroller tipping over, damaging the product and injuring the user.

## 17. OVERCOMING THRESHOLDS OR CURBS

When overcoming small obstacles (small thresholds, small curbs, etc.), use your foot to lock the tilt foot and push handle down and toward you to lift the front wheels of the stroller and carefully drive the front wheels over the obstacle. Then slowly pull the push handle up, away from the ground, to lift the rear wheels of the stroller and carefully drive the rear wheels over the obstacle.

## 18. TRANSPORT IN THE CAR AND IN OTHER VEHICLES

- Remove all accessories from the stroller and store them in a safe place. Fold the stroller and place it in the trunk.

If the dismantled stroller is not in the trunk of the car separate from the passenger compartment, the stroller frame must be carefully secured to the vehicle. For this purpose, you can use unused seat belts.

- In order to transport a stroller in other vehicles, such as trains, trams, buses, planes, ships, etc., it is necessary to ask the persons in charge of transportation in these means of transport about the conditions for safe travel to the destination with a stroller and relevant requirements.



**The MEWA stroller should not be used as a seat to transport an user in a vehicle.**

## 19. CLEANING AND CARE

The upholstery can only be washed by hand at up to 30° C.

Stains should be removed as soon as they form on the surface of the stroller.

**ATTENTION:** Do not clean with chemicals containing solvents and bleach. The manufacturer is not responsible for damages caused by the use of unsuitable cleaning agents.

Make sure the product is always dry before use.

If using detergents dedicated to this type of material - follow the instructions of the detergent used.



Hand wash only



Do not bleach



Do not tumble dry  
drum



Dry in position  
horizontal



Do not iron

## 20. STORAGE OF THE STROLLER

Clean stroller should be stored in a clean, dry and well-ventilated room, preferably at 0-35°C. For long- term storage, additionally follow the steps described in point 21 "every one month".

## 21. SERVICE AND PERIODIC INSPECTIONS

For safe and trouble-free use of the stroller, it is necessary to regularly and thoroughly submit it to the inspections and servicing described below.

In a situation where the stroller user is not able to carry out the above-mentioned activities on their own, they should return the stroller to a specialized stroller service point or directly to the Manufacturer, who will properly carry out the inspection and necessary servicing activities. These activities may be subject to an additional fee.

**NOTE:** The Manufacturer is responsible for defects or technical damage to the stroller caused only when the stroller is operated correctly. Responsibility for the proper operation of the stroller rests with the Manufacturer only if all recommendations of the Manufacturer are strictly followed.

**NOTE:** The Manufacturer is responsible for the proper functioning of the stroller only if original parts and original accessories are used in it.

All repairs should be carried out in specialized authorized Services (see point 24) or directly at the Manufacturer. Repairs to the stroller performed by anyone other than an authorized Service or the Manufacturer will void the warranty. The Manufacturer is not responsible for injury or damage of the product that may be a consequence of repairs performed by persons other than the authorized Service or the Manufacturer.

### Service should be performed:

- **Each time before using the stroller**
  - Check the efficiency of the central brake - After activating the central brake, both rear wheels of the stroller should be locked and cannot rotate.
  - Check the attachment of structural components - Inspect all mechanisms of the stroller and its moving parts, and in particular, pay attention to: correct mounting of the backrest to the frame of the stroller, proper mounting of the wheels, proper functioning of the five-point belts, correct functioning of the seat angle adjustment mechanism, correct functioning of the backrest recline mechanism and proper mounting of the footrest.
  - Check the condition of the tires - Check that the tires on all four wheels are not damaged or worn.
  - Inspection of solid frame parts, seat with backrest and footrest frame - Visually inspect the condition of the structural profiles. These components must not be damaged or cracked.
- **Every one month**
  - Cleaning of the stroller
  - Visual check of all the moving elements, bearings, adjusting mechanisms – lubricate with silicone grease.
  - Check all bolted connections for looseness.

## 22. UTILIZATION

Disposal and recycling: in accordance with country-specific regulations.

## 23. COMPLIANCE WITH THE REQUIREMENTS FOR MEDICAL DEVICES

Class I medical device in accordance with the Regulation of the European Parliament and of the Council (EU) 2017/745 of April 5, 2017 on medical devices. The manufacturer issued an appropriate declaration of conformity for the product, which is available at [www.mywam.eu](http://www.mywam.eu)

## 24. LIST OF AUTHORISED SERVICE POINTS

POLAND

MyWam Sp. z o.o.

Wiosny Ludów 91, 40-373 Katowice, Poland

tel: +48 32 733 11 31

website: [www.mywam.eu](http://www.mywam.eu)

email address: [mywam@mywam.eu](mailto:mywam@mywam.eu)

## 25. WARRANTY CARD

Series and number: .....

Purchase date: ..... - ..... - .....

Seller stamp and signature:

.....  
.....  
.....

### Warranty conditions

- The Manufacturer (Guarantor) provides the Buyer with a warranty for the purchased stroller for a period of 24 months, counting from the date of purchase.
- **The buyer complaining about the stroller should deliver the stroller to the Seller in a package, clean (free of stains, additional equipment not belonging to the stroller, moisture, hair, etc.) together with the elements of the stroller protected against damage.**
- **Damage caused by insufficient protection of the stroller shipment and its components is not subject to warranty repairs.**
- The complainant (Buyer) may not demand that the Manufacturer repair the damage in the event of damage to the shipment if he incorrectly labels or incorrectly insures the shipment of the stroller and its components.
- The warranty period in the case of accepting the complaint is extended by the time of repair of the stroller, which is counted from the moment of handing over the stroller to the Seller under warranty until the day on which the stroller was ready to be collected.
- The activities provided for in the instruction, which the stroller user is obliged to perform on his own and at his own expense, cannot be performed as part of the warranty repair.
- Failure to use the stroller in accordance with the intended use, inconsistent with the instructions for use or inconsistent with the principles of proper operation will void the warranty.
- Any structural changes to the stroller void the warranty (in the area of change or in its entirety).
- Damage to the stroller caused by: improper operation, improper maintenance, improper storage of the stroller, mechanical damage and natural abrasion and wear of parts (e.g. upholstery, wheels) are not covered by the warranty.
- The manufacturer warns against warranty repairs in unauthorized service facilities. Finding such a repair or unauthorized modification of the stroller will void the warranty.
- **The warranty card is valid provided that it has: the date of sale confirmed by the Seller's company stamp, the Seller's signature, the type and serial number of the stroller.**
- The warranty card is valid provided that it does not contain deletions or corrections.
- The manufacturer of the stroller conducts a paid post-warranty service at its headquarters.
- The manufacturer specifies the expected service life of the product at 4 years. Further use after this period is possible, but taking into account the care for safety and comfort of use, we recommend that the manufacturer or authorized service center carry out a technical inspection. The inspection service is payable and includes an assessment of the technical condition of the device, safety verification and any necessary maintenance activities.
- In matters not covered by this warranty card, the provisions of the Civil Code (Poland) shall apply.
- All matters related to claims under the warranty or warranty of the stroller should be directed to the Seller.

## 26. NOTES