



ZEFIR

SPECIAL NEEDS STROLLER FOR CHILDREN

NEW

GENERAL INFORMATION

The **ZEFIR** special needs stroller is a light and compact model dedicated to children between **80 and 140 cm** and maximum body weight **up to 40 kg**.

ZEFIR has many useful functions, such as the ability to recline the backrest and adjust the height of the footrest. It is also equipped with the "**I grow with you**" function that allows you to increase **the backrest height** (by mounting the extension tubes) as well as increase **the seat depth** by using the roll-up mechanism



  **MEDICAL DEVICE - CLASS I**

Like all our strollers, **ZEFIR** has a wide range of additional accessories that facilitate the organization of daily walks and are useful in less favorable weather conditions.

THIS IS MEDICAL DEVICE.
USE IT IN ACCORDANCE WITH THE INSTRUCTIONS FOR USE OR THE LABEL.

COLOR

white frame / glossy



black frame / matt



INDICATIONS

- paralysis and paresis of lower limbs various origins
- paralysis and paresis of the trunk
- weakness of postural muscles, responsible for maintenance correct body position
- cerebral palsy
- myelomeningocele
- muscular dystrophies

CONTRAINDICATIONS

In case of contraindications to a sitting position and/or passive standing position, adjustment of the stroller should be consulted with a physiotherapist or attending physician.



Zefir wheels come in 2 options: slim and off-road. The first one (1) offers a sleek design, while the latter (2) provides a classic look and the necessary support on uneven terrain.



Keep all accessories attached and fold the stroller in a flash! Enjoy the light and minimalist design all around.



THE DESIGN OF THE SPECIAL NEEDS STROLLER ZEFIR ALLOWS ITS SIZE TO BE ADAPTED TO THE CHILD.



MINIMUM SETTINGS



MAXIMUM SETTINGS



FOOTREST HEIGHT ADJUSTMENT

Thanks to a simple mechanism of detaching the footrest, we can lower it to **40 cm** down from the seat level. The seat roll-up mechanism allows to increase its surface by **6 cm**, giving us a maximum depth of **30 cm**.

The backrest height can be increased by **10 cm** using the extension pipes (**maximum height 60 cm**).

If need be, the seat **width can be reduced by 9 cm** using narrowing inserts.



ADJUSTMENT OF THE BACKREST
RECLINE



EQUIPMENT



**NARROWING
INSERTS**



**BACKREST
EXTENSION**



SAFETY BARRIER



ABDUCTION BELTS



FIVE-POINTS BELTS



**ADJUSTABLE
ABDUCTION BLOCK**



THERAPY TRAY



**CANOPY WITH
WINDOW**



HEADREST



TRAVEL BAG



SHOPPING BASKET



SUN UMBRELLA



WINTER GLOVES



RAIN COVER



SLEEPING BAG



MOSQUITO NET

TECHNICAL PARAMETERS OF THE STROLLER

seat width: 29*/38 cm

effective seat depth: 24/30 cm

backrest height: 50/60 cm

backrest width: 29*/37 cm

distance between footplate and seat: 20, 30, 40 cm, 3 positions

*values are given with inserts

total weight: 13 – 17 kg**

stroller length: 95 – 113 cm**

stroller height: 102 – 122 cm**

stroller width: 62 cm**

**values with universal wheels

IMPORTANT NOTE! The choice of a medical device appropriate for a specific disease entity and condition of the patient must be made with the help of a specialist doctor or physiotherapist. Before use, read the operating manual and the label or consult your doctor, as this medical device may not be suitable for you.